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Please note: To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

- Position the impression tray in the mouth within the working time and allow the impression to form.
- Remove the impression from the mouth once the setting time has ended.

Removal of the impression

- Remove the impression parallel to the longitudinal axis of the prepared teeth.
- After removal, proceed as described in the section »Cleaning and disinfection of the final impression«.

Cleaning and disinfection of the final impression

- Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.
- Follow the instructions of the disinfectant solution’s manufacturer precisely.
- Leave the final impression to dry and store until casting, see »Storage of the impression«.
- For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0, www.dgzmk.de.
- ADA recommends following Council on Dental Materials, Instruments and Equipment. Disinfection of Impressions. JADA 1991;122(3):110, www.ada.org

Casting and galvanization

- The impression should be cast 30 minutes at the earliest after removal from the mouth.
- DMG recommends using special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 6873, Type 4, e.g. Fujirock II, GC). The impression can be copper or silver-plated.

Storage of the impression

- Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place!
- Protect against exposure to direct sunlight!

The impression is stable and can be cast even after months without loss of accuracy.

Technical data

Silagum-Automix Mono	
Working time*	≤ 1:30 min
Recommended time in mouth	≥ 2:30 min
Dimensional change after 24 h	≤ 0.5%
Compression rest	≤ 0.5%
Strain in compression	≤ 3.5%

Silagum-Handmix Mono	
Mixing time	0:30 min
Working time*	≤ 2:30 min
Recommended time in mouth	≥ 3:30 min
Dimensional change after 24 h	≤ 0.35%
Compression rest	≤ 0.4%
Strain in compression	≤ 4.0%

*The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase these times.

Disinfection of the Automix dispenser

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

Storage and shelf life

- Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place!
- Do not store in the fridge or freezer.
- Protect against exposure to direct sunlight.
- After the application, leave the used tip on the cartridge to cap unused material.
- Do not use after the expiration date.

Composition

Addition curing vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

Classification

ISO 4823:2000, type 2: medium-bodied consistency

Available product sizes

Refill packs	
2 cartridges @ 50 ml paste, 4 Automix tips, 4 Intraoral tips	REF 909718

Economy pack	
8 cartridges @ 50 ml paste, 16 Automix tips, 16 Intraoral tips	REF 909719
8 tubes @ 135 ml (170 g) paste, 2 mixing blocks, 2 spatulas	REF 909427

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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