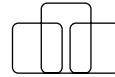
SILECT®

müller-
omicron
DENTAL

- DE Gebrauchsanweisung
EN Instructions for use
FR Mode d'emploi
ES Instrucciones de uso
IT Istruzioni per l'uso
RO Instrucțiuni de utilizare
PT Manual de instruções
EL Οδηγίες χρήσης
PL Instrukcja stosowania

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Premium impression material, C-silicone / polysiloxane

Thank you for the confidence you have shown in our company. You have made the right decision with the SILECT® premium impression system. Please be sure to read these instructions for use thoroughly before starting to use the product! SILECT® products are only intended for use in dental applications by correspondingly qualified professionals. The medical device must be employed responsibly and with due caution.

Indication

Crowns and bridges, partial crowns, inlays, onlays, complete and partial dentures.

Impression technique

Can be used for the putty-wash technique, double mixing technique, situation impressions and relining.

Dosage / processing (see also illustration)

SILECT® putty + SILECT® catalyst:

Remove the required quantity of putty and catalyst in the correct dosage ratio as shown in the illustration. Draw out the putty and catalyst to create a small strand, then fold it together again and knead it with your fingertips. Repeat this process a few times until you achieve a homogeneous colour, but always complying with the minimum mixing time of 30 seconds. Close containers and tubes again immediately after use!

SILECT® Light + SILECT® catalyst

Place equal length strands of SILECT® Light and SILECT® catalyst side by side in the correct dosage ratio as shown in the illustration on a mixing block or glass plate. Always ensure an even strand diameter. Pick up the catalyst with a spatula, distribute it evenly over the other component and mix thoroughly with the spatula for 30 seconds until you achieve a homogeneous colour.

Mixing ratio

Adding too much or too little catalyst can affect the total setting time. As a general rule, adding more catalyst accelerates the setting process, while adding less catalyst slows it down. It is possible to increase or reduce the amount of catalyst by +/- 25%.

Taking the impression

When using a suitable tray adhesive, please follow the manufacturer's instructions for use. After inserting the tray with the SILECT® materials into the patient's mouth, press it into place briefly and then hold in place until fully cured. Always check the material intraorally to assess the degree of curing. When using the putty-wash technique (2-step), the first impression must be cleaned and dried in order to ensure cross-linking with the wash material.

Disinfection

The impressions can be disinfected with a 2% glutaraldehyde solution. After removing the impression from the mouth, rinse it under running water for 15 seconds. Then disinfect the impression by immersing it in the disinfectant thoroughly. Observing the disinfectant manufacturer's specifications is essential. We recommend the (aldehyde-free) products Dentoprint MD liquid and Dentoprint MD pur from Müller-Omicron GmbH & Co. KG. Further information can be found at www.dgzmk.de, and in J. Prosthet. Dent. 1999 May, 81(5), 621.

Electroplating:

The impressions can be electroplated with copper or silver in standard baths.

Model fabrication

The impression can be poured after just 30 mins and for up to 10 days. All class 3 and 4 gypsums which comply with the requirements in ISO 6873, e.g., Moldano, Kulzer (Type 3) and Fujirock EP, GC (Type 4), can be used. Undamaged impressions can be poured several times.

Working and safety instructions

The products described above must only be used in accordance with their instructions for use. The user has sole liability for any other use which contravenes these instructions for use.

- Close containers and tubes tightly immediately after use and do not interchange the lids and measuring spoons.
- Solutions used prior to impression taking, such as mouth rinses or retraction solutions, must be removed thoroughly.
- Do not leave any impression material residue in the mouth.
- Avoid contact with clothing, as cross-linked silicones are chemically resistant and leave stains which cannot be removed.
- Do not use SILECT® products together with polyether, polysulphides or addition-curing silicones.
- To prevent possible incompatibilities, do not combine SILECT® products with C-silicones from other manufacturers.
- Storage: Store in a dry place at between 5°C and 27°C (41°F and 80.6°F).
- Shelf life: See date printed on package / label. Do not use after expiry date.
- Working temperature: Room temperature between 18°C and 25°C (64.4°F to 77°F).

Contraindication:

The product must not be used in patients with known allergies to flavourings or other ingredients (see safety data sheet or ask the manufacturer). This should be clarified before use!

This product should not be used in pregnant women, breastfeeding mothers or children under six years of age. Take the impression with an A-silicone instead, e.g., betasil VARIO. This product may only be used by dental professionals.

Warnings for the SILECT® catalyst

Inhalation: Take person out into the fresh air, administer oxygen if necessary.

Contact with skin: Avoid prolonged contact with the skin. Following contact, wash the skin thoroughly with soap and water. We recommend wearing suitable protective clothing and gloves.

General warnings

Eye contact: Avoid contact with eyes. In case of contact, rinse eyes immediately with plenty of clean water and consult an ophthalmologist without delay.

Swallowing: If swallowed, rinse out the mouth and drink plenty of water. Always contact a doctor.

Allergic reactions: It is not possible to rule out allergic reactions in hypersensitive people. If acute allergic reactions should occur during the treatment, treatment with the product should be stopped immediately and a doctor should be consulted. In case of known allergies, we generally recommend taking the impression with an A-silicone, e.g., betasil VARIO from Müller-Omicron GmbH & Co. KG.

Should a serious incident occur unexpectedly in connection with this medical device, it must be reported to the manufacturer and the competent authority in your Member State.

Please also observe the warnings in the safety data sheet!

More information at www.mueller-omicon.com

Errors and omissions excepted.

Contains hazardous substances / Tris(2-methoxyethoxy)vinylsilane

Technical data and product features

Product	Type (EN ISO 4823)	Colour	Working time incl. mixing time*	Time in mouth	Deformation under pressure (approx.)	Recovery from deformation (approx.)	Linear dimensional change (approx.)	Flavour
SILECT® putty	Type 0	Mint green	1 min. 30 sec.	≥ 3 min. 30 sec.	1.85%	98.90%	0.10%	Mint fresh
SILECT® light	Type 3	Blue-Violet	1 min. 30 sec.	≥ 3 min. 30 sec.	5.30%	99.00%	0.50%	Neutral

* At 23°C / 73.4°F. The times specified in the table above are based on a relative humidity of 50 +/- 10%. As a general rule: higher temperatures accelerate setting and lower temperatures delay it.

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