
SmartLite® Focus® Pen-Style LED Curing Light

Lampe LED à photopolymériser miniaturisée
Lampada fotopolimerizzatrice LED modello penna
“Pen-Style” LED Polymerisationslampe
Lámpara de polimerización de LEDs tipo lápiz
Fotopolimerizador LED estilo caneta
Ledpolymerisatielamp in penvorm
LED-härdljuslampa i pennformat
Penneformat LED hærdelampe
Lampa polimeryzacyjna LED typu „pen”
Polymerační LED lampa v podobě pera
Ceruzková polymerizačná lampa vybavená LED
Toll formájú LED polimerizációs lámpa
Lampă de polimerizare cu LED, în formă de stilou
Llambë polimerizuese LED model lapsi
Συσκευή πολυμερισμού LED τύπου πέννας
Светодиодная лампа в форме ручки

SmartLite® Focus® curing light 1

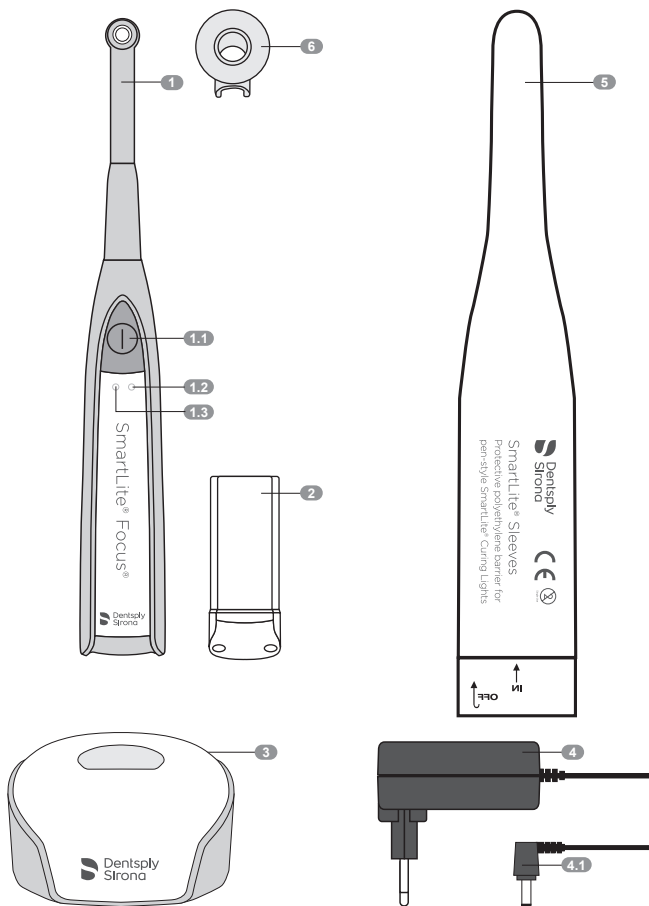
- 1 SmartLite® Focus® curing light
- 1.1 ON/OFF key
- 1.2 and 1.3 Battery lights
- 2 SmartLite® Focus® battery pack
- 3 SmartLite® Focus® charging base


- 4 Power connector incl. adapters
- 4.1 Jack
- 5 SmartLite® Sleeves
- 6 SmartLite® Focus® Shields

SmartLite® Focus® curing light – operation at a glance

Procedure A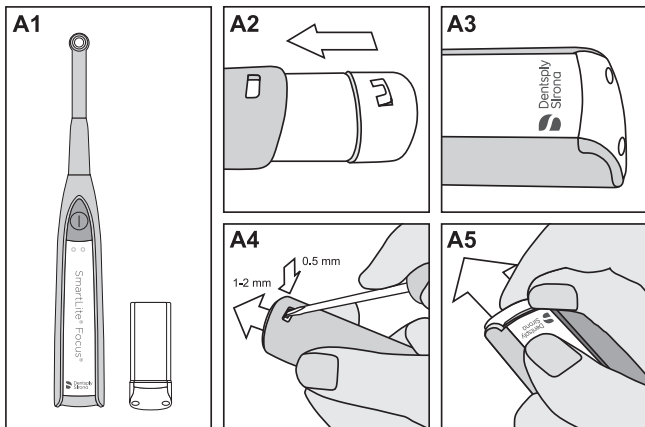

Procedure B

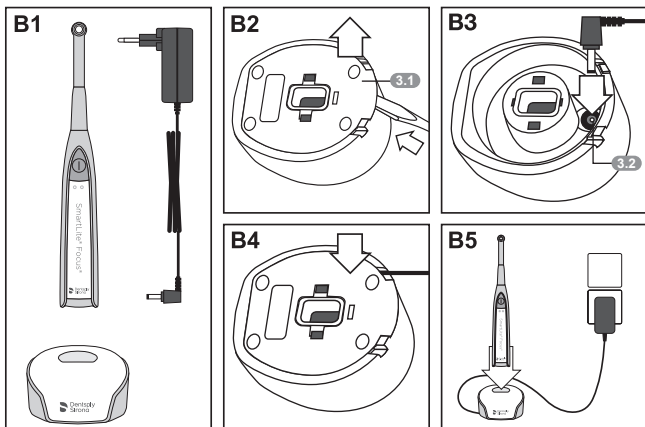
3.1 Floor panel


3.2 Charging base socket

SmartLite® Focus® curing light – operation at a glance

Procedure C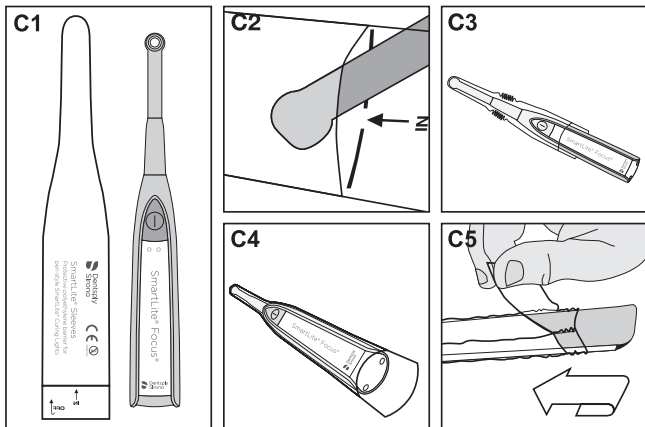

Procedure D

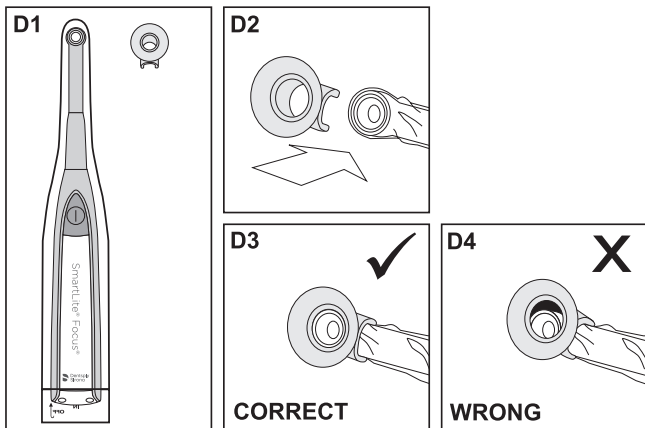Instructions for Use	English	2
Mode d'emploi	Français	16
Istruzioni per l'uso	Italiano	30
Gebrauchsanweisung	Deutsch	44
Instrucciones de uso	Español	58
Instruções de utilização	Português	72
Gebruiksaanwijzing	Nederlands	86
Bruksanvisning	Svenska	100
Brugsanvisning	Dansk	114
Instrukcja użycia	Polski	128
Návod k použití	Český	142
Návod na použitie	Slovenský	156
Használati utasítás	Magyar	170
Instrucțiuni de utilizare	Română	184
Udhëzime përdorimi	Shqip	198
Οδηγίες χρήσης	Ελληνικά	212
Инструкция по эксплуатации	Русский	226

SmartLite[®] Focus[®]

Pen-Style LED Curing Light

CAUTION: For dental use only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	4
3 Step-by-step instructions	6
4 Hygiene and maintenance	7
5 Maintenance	12
6 Reorder information, technical data, warranty terms	13
7 Correspondence	15
A. Addendum SmartLite[®] Focus[®]	241

1 Product description

The **SmartLite[®] Focus[®]** curing light is a cordless pen-style, LED light polymerization device for use by trained dental professionals in dental offices or dental laboratories.

SmartLite[®] Focus[®] curing light is characterized by:

- Small size and lightweight ergonomic design.
- Compact cordless design with convenient handling features and exchangeable battery pack.
- Smart battery charging mechanism with virtually permanent light operating availability.
- Individually adjustable LED tip, rotatable by 330°.
- LED tip design providing excellent intraoral access.
- Polymerization area (optic effective cross-sectional area) of minimum 7.5 mm in diameter.
- Up to 20 seconds curing time per activation with audible signal every 10 seconds.
- Advanced heat management system limiting LED tip temperature.

1.1 Indications

- Light curing of dental materials which cure at a wavelength of 460-490 nm.

1.2 Contraindications

- Use in patients prone to photobiological reactions (including patients with solar urticaria or erythropoietic protoporphyria) or those currently undergoing treatment with photosensitizing pharmaceuticals.

1.3 SmartLite® Focus® curing light ① – operation at a glance

1.3.1 ON/OFF key ①.①

- Starts or disrupts the 20-second curing cycle.

1.3.2 Battery light

- **Permanently green ①.②** (only indicated when SmartLite® Focus® curing light handpiece is inserted into the charging base ③): SmartLite® Focus® curing light is fully recharged.
- **Flashing green ①.②** (only indicated when SmartLite® Focus® curing light handpiece is inserted into the charging base ③): SmartLite® Focus® curing light is in Standard Charging Mode.
- **Fast flashing green ①.②** (only indicated when SmartLite® Focus® curing light handpiece is inserted into the charging base ③): SmartLite® Focus® curing light is in Automatic Quick Charging Mode.
- **Flashing yellow ①.③**: SmartLite® Focus® is almost discharged and needs recharging. At first occurrence approximately 1 minute curing time left.
- **Permanently yellow ①.③**: SmartLite® Focus® curing light is fully discharged (no curing time left) and needs recharging.

1.3.3 Audible signals

- **One blip**: At start of curing cycle, after 10 seconds curing time together with flashing blue LED, and when SmartLite® Focus® curing light is fully recharged.
- **Two blips**: After 20 seconds consecutive curing time. Curing light automatically extinguishes after 20 seconds.
- **Repetitive short blip every 7 seconds together with flashing yellow battery light**: Only occurs when device is turned off. Approximately 1 minute operation time left at first occurrence. Device needs recharging.
- **One long blip together with permanent yellow battery light**: Only occurs when device is turned off. Device is fully discharged and needs recharging.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

Never modify SmartLite® Focus® or any of its equipment. Any modification may compromise safety and effectiveness.

2.1.1 SmartLite® Focus® handpiece 1

- Always make sure SmartLite® Focus® Shields 6 are securely fixed to SmartLite® Focus® to avoid accidental aspiration (press SmartLite® Focus® Shield firmly into correct position; see Procedure D for details).
- Always make sure light aperture is not covered by SmartLite® Focus® Shields (see Figure D3 and Figure D4).
- Do not try to remove LED tip from handpiece as they are permanently connected.
- The LED tip is rotatable by 330°. Do not attempt to fully rotate the LED tip.
- Do not use the device as a tissue retractor, as this may damage the connection between LED tip and handpiece.
- Do not use a damaged device, e.g., if, among others the glass cover of the LED tip is scratched, broken or missing.
- The SmartLite® Focus® Shields will wear over time. Exchange defect shield against spare shields included in the package or available as refill (see also [6.1 Accessories]).
- Only authorized technicians should repair the handpiece or battery pack.

2.1.2 Charging base 3

- Low voltages are present inside the charging base (12 V DC). Do not open charging base when connected to power source. Use only under dry conditions. Do not use if the charging base or handpiece is wet. Avoid short circuit between contact pins inside the charging base. Only authorized technicians should repair the charging base.
- Do not use for voltages different from the range indicated on the charging base.
- Note: Unplug power connector 4 from power source before disconnecting power connector from the charging base.
- Always make sure the charging base is placed offside the dental unit and only touched with clean/disinfected gloves to prevent from exposure to spatter or spray of body fluids.
- Always make sure handpiece is completely reprocessed and thoroughly dry before inserting it into the charging base.

2.1.3 Battery pack 2

Prevent battery pack from short circuit during use and storage.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Anyone with a history of retinopathy should consult their eye specialist before operating this unit. Use the SmartLite® Focus® curing light extremely carefully and comply with all the necessary safety precautions (including wearing suitable, light filtering safety glasses).
- Anyone who has had a cataract operation may be especially sensitive to light and should be advised against undergoing treatment with the SmartLite® Focus® curing light unless adequate safety precautions are taken such as wearing suitable, light filtering safety glasses.
- Do not use a SmartLite® Focus® curing light which has not been properly reprocessed. Protect SmartLite® Focus® curing light from gross contamination by applying the single use SmartLite® Sleeves. SmartLite® Sleeves are intended for single use only. Discard after use. Do not reuse sleeves in other patients in order to prevent cross-contamination.
- For electrical medical devices specific safety precautions apply which have to be observed.¹ Do not use mobile phones or other portable communication tools, as they could interfere with the function of the SmartLite® Focus® curing light.
- Never aim the light directly at unprotected soft tissues, as this may cause injury or irritation. Do not aim the light at eyes. Light reflected from the tooth surface may also injure eyes. Use the SmartLite® Focus® Shields supplied with the unit or suitable, light filtering safety glasses.²
- Limit the action of the light to the area being treated.
- All dental curing lights cause a certain degree of heat development. Extended operation in areas near the pulp and soft tissues may result in severe damage. Do not cure the same tooth surface for more than 20 seconds at a time. If required, polymerize at intermitting intervals of 20 seconds each.
- Use of any accessories other than specified in this Instructions for Use may result in increased emissions or decreased immunity of the SmartLite® Focus® and its components.
- Do not use adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, closely watch SmartLite® Focus® and its components to verify normal operation in this configuration.

¹ For technical details (electromagnetic emissions and electromagnetic immunity) see: www.dentsply.eu – chapter: IFU/MSDS.

² SmartLite® Focus® is classified as a Risk Group 2 device according to EN 62471.

3 Step-by-step instructions

3.1 Installation and charging (Procedure A and Procedure B)

1. Remove floor panel 3.1 of charging base by lifting floor panel at its back end (see Figure B2).
2. Connect jack 4.1 to charging base socket 3.2 (see Figure B3).
3. Close floor panel (see Figure B4) and plug in power connector 4.
4. Make sure power socket used for power connector is accessible at all times in case of necessary emergency disconnection.
5. Insert battery pack 2 into SmartLite® Focus® handpiece 1 (see Figure A2 and Figure A3).
SmartLite® Focus® battery pack is pre-charged and ready to use upon purchase.
6. To recharge the battery, insert the handpiece 1 into the charging base 3 (see Figure B5). The battery light will be flashing green, indicating battery is recharging.

The SmartLite® Focus® Automatic Quick Charging Mode allows a minimum of 5 minutes operating time after 10 minutes recharging.

3.2 Operation

1. Protect LED tip from gross contamination by applying the single use SmartLite® Sleeves (see Procedure C).
2. Use the SmartLite® Focus® Shields supplied with the unit in combination with mounted SmartLite® Sleeves or use suitable, light filtering safety glasses (see Procedure D). Always make sure SmartLite® Focus® Shields are securely fixed to SmartLite® Focus® to avoid accidental aspiration (press SmartLite® Focus® Shield firmly into correct position). Always make sure SmartLite® Focus® Shields are properly mounted on SmartLite® Focus® without covering the light aperture (see Figure D3 and Figure D4).
3. Adjust the LED tip: The LED tip is rotatable by 330°, thus the position of light emission may be individually adjusted. The LED tip must be adjusted to an angle that allows optimal curing of the restoration. The LED tip should be positioned **as closely as possible to the restoration**.
4. Polymerization: Briefly press the ON/OFF key [1.3.1 ON/OFF key]. Audible operation signals are described in [1.3.3 Audible signals].
5. To stop curing light before the end of the 20 second cycle, press ON/OFF key at any time. To cure a restorative material that requires a curing time longer than 20 seconds, repeat activation by pressing the ON/OFF key after the end of each 20 second curing cycle (see also [2.2 Precautions]).
6. Curing times for a range of Dentsply Sirona materials are outlined in the Curing Guide provided. When using other products, please refer to respective product's Instructions for Use.

4 Hygiene and maintenance

Cross-contamination.

Infection.

- Do not reuse single use products. Dispose of in accordance with local regulations.
- The barrier is designed for single use and must be disposed of after each use in accordance with local regulations. The barrier is not a replacement for cleaning, disinfection, and sterilization.
- Reprocess reusable products as described below.

NOTICE

Wrong cleaning or disinfection method.

Damage to device.

1. Do not submerge into liquids.
2. Do not clean or disinfect with solutions based on orange oil, acetone, chlorine bleach/sodium hypochlorite or hydrogen peroxide. Do not use the following products: Optim 33 TB, CaviCide1, Discide Ultra, CaviWipes.
3. Removing of the battery pack is not recommended for regular cleaning and disinfection.
4. Start reprocessing within 1 hour after use.
5. Ensure handpiece and charging base are fully dry before inserting handpiece into charging base.

For Manual Cleaning and Disinfection:

1. Discard used gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.
3. Use a new clean pair of examination gloves.

Cross-contamination.

Infection.

- Always make sure the handpiece is completely reprocessed before inserting it into the charging base.
- Charging base should be reprocessed as soon as reasonably practical after being exposed to spatter or spray of body fluids, or touched by contaminated hands or contaminated light handpiece.

Electrical short-circuit or dangerous malfunction.

Injury.

- Safeguard handpiece against liquid penetration during cleaning and disinfection.
- Do not try to remove LED tip from handpiece as they are permanently connected.

Instructions for Cleaning and Disinfecting SmartLite® Focus® Light Handpiece, Charging Base	
Warnings	<ul style="list-style-type: none"> • The SmartLite® Focus® Shield should be removed and cleaned/disinfected/sterilized as outlined below. The SmartLite® Focus® light handpiece and charging base are not sterilizable by autoclave. • Intermediate-level disinfection is appropriate for the handpiece and charging base. • Do not autoclave in steam autoclave. • Do not clean/disinfect in automated washer/disinfector. • Do not immerse in liquid. • Disconnect the power supply plug from the power outlet and charging base unit prior to cleaning/disinfection.
Limitations on reprocessing	<ul style="list-style-type: none"> • Repeated reprocessing has minimum effect on these instruments. End of life is normally determined by wear and damage due to use. • Device can be reprocessed for up to 6000 times. • Cold liquid immersion disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.
Point of use	<ul style="list-style-type: none"> • Remove protective barrier and discard according to local regulations. • Remove excess soil with disposable cloth/paper wipe in combination with an alcohol-based, tuberculocidal, quaternary ammonium solution with a label claim for cleaning e.g., Volo-Wipes® disinfecting/cleaning/deodorizing Wipes or pH-neutral, phosphate-free cleaning solution (e.g., Dr. Schumacher Instru Plus [3%]). • Remove SmartLite® Focus® Light Shield. Reprocess as outlined below. • Do not remove battery pack. Do not attempt disassembly of light handpiece or charging base. • It is recommended that the device be reprocessed as soon as is reasonably practical following use. • Start reprocessing within 1 hour after use. • Charging base should be reprocessed as soon as reasonably practical after being exposed to spatter or spray of body fluids, or touched by contaminated hands or contaminated light handpiece.
Containment/ Transportation	No particular requirements. Handle with care.

Cleaning and Disinfection: Automated	Do not use automated washer/disinfectors for reprocessing SmartLite® Focus® light handpiece or charging base. Component damage will occur.
Cleaning: Manual	<ol style="list-style-type: none"> 1. The SmartLite® Focus® light handpiece and charging base have to be cleaned by scrubbing with a pre-impregnated wipe or disposable towel soaked with an alcohol-based, tuberculocidal, quaternary ammonium solution with a label claim for cleaning (e.g., VoloWipes disinfecting/cleaning/deodorizing Wipes) or pH-neutral, phosphate-free cleaning solution (e.g., Dr. Schumacher Instru Plus [3%]) according to local regulations and use according to cleaning solution manufacturer's Instructions for Use until it is free of visible residues. Remove all visible soil, ensuring fluid penetrates all crevices. Do not allow solution to penetrate the casing. Discard used wipes. Additional wipes may be used. 2. Allow the device to air dry.
Disinfection: Manual (Intermediate-Level)	<ol style="list-style-type: none"> 1. After cleaning wipe all device surfaces with a new single use cloth in combination with an alcohol-based, tuberculocidal, quaternary ammonium solution e.g.: VoloWipes disinfecting/cleaning/deodorizing Wipes, BBraun Meliseptol® Rapid, approved according to local regulations, and use according to disinfectant solution manufacturer's Instructions for Use. Pay special attention to seams, areas around buttons, window, and crevices. 2. Allow the device to air dry. 3. Wipe the devices with a sterile, clean, lint-free cloth dampened with distilled water. 4. Pay special attention to device seams and insertions.
Packaging	No particular requirements.
Sterilization	Sterilization is not allowed. No methods have been validated. Do not subject components to Steam autoclaving or liquid chemical sterilant immersion. Component damage will occur.
Drying	Wipe the devices dry with a sterile, clean, lint-free cloth. Allow the components to fully air dry before storage.
Maintenance	<p>Visually inspect to ensure that all contamination has been removed.</p> <p>Visually inspect power supply and cord for damage.</p> <p>Components that are damaged, worn, or distorted should be discarded.</p> <p>No additional maintenance or lubrication is recommended.</p>
Storage	Store the SmartLite® Focus® light handpiece and charging base at room temperature, away from moisture or excessive humidity.
Manufacturer Contact	Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.

Instructions for Cleaning, Disinfecting and Sterilizing SmartLite® Focus® Light Shield	
Warnings	<ul style="list-style-type: none"> • These instructions are for use ONLY for the light shield. The handpiece and charging base should be disinfected according to the procedures in the "Instructions for Cleaning and Disinfecting SmartLite® Focus® Light Handpiece, Charging Base" section above. • The SmartLite® Focus® Shield should be removed and cleaned/disinfected/sterilized as outlined below. The SmartLite® Focus® Light handpiece and charging base are not sterilizable by autoclave. • Steam autoclaving sterilization is appropriate for the light shield. • Do not allow the device to exceed 137 °C/279 °F.
Limitations on reprocessing	<ul style="list-style-type: none"> • Repeated reprocessing has minimum effect on these instruments. End of life is normally determined by wear and damage due to use. • Device can be reprocessed for up to 60 times. • Cold liquid immersion disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use.
Point of use	<ul style="list-style-type: none"> • Remove protective barrier and discard according to local regulations. • Remove excess soil with disposable cloth/paper wipe in combination with a pH-neutral, phosphate-free cleaning solution (e.g., Dr. Schumacher Instru Plus [3%]). • Remove SmartLite® Focus® light shield. Reprocess as outlined below. Reprocess handpiece and charging base as outlined in the "Instructions for Cleaning and Disinfecting SmartLite® Focus® Light Handpiece, Charging Base" section above. • It is recommended that the device be reprocessed as soon as is reasonably practical following use. • Start reprocessing within 1 hour after use.
Containment/Transportation	No particular requirements. Handle with care.
Cleaning and Disinfection: Automated	<p>Thermal disinfecter: Manufacturer's specification according to DIN EN ISO 15883. Cleaning program (A0 value > 600 or, at least 1 min at 90 °C/194 °F) as indicated by the manufacturer in the operating instructions.</p> <p>Follow manufacturer's recommendation for use of detergent and neutralizer, observing concentrations and contact times.</p>
Cleaning: Manual	<p>As an alternative to automated cleaning and disinfection, the SmartLite® Focus® Shield has to be cleaned by scrubbing with hot water and a pH-neutral, phosphate-free cleaning solution (e.g., Dr. Schumacher Instru Plus [3%]).</p> <ol style="list-style-type: none"> 1. Immerse SmartLite® Focus® Shield in the detergent solution. Clean with a soft brush. 2. Rinse under running potable water. 3. Dry with a lint-free single use cloth.

Disinfection: Manual (Inter- mediate-Level)	<ol style="list-style-type: none"> 1. After cleaning wipe all device surfaces with a single use cloth in combination with an alcohol-based, tuberculocidal, quaternary ammonium solution (e.g., BBraun Melisepto® Rapid). 2. Thoroughly disinfect the surface with the soaked cloth, paying special attention to any cavities and holes. Ensure the entire surface remains wet for the entirety of the required contact time in accordance with the disinfectant solution manufacturer's Instructions for Use. 3. Rinse for 30 seconds under running potable water. 4. Dry with a lint-free single use cloth.
Packaging	Paper/plastic steam sterilization pouches (e.g., AssurePlus® Sterilization Pouches) may be used, but are not required.
Sterilization	<p>After Cleaning and Disinfecting, steam autoclaving is required. Prevacuum Steam Sterilization:</p> <ul style="list-style-type: none"> • Full Cycle: 134 °C/274 °F for 3 minutes 30 seconds. <p>Follow manufacturer's instructions for loading and operation cycle.</p>
Drying	Use the drying cycle of the autoclave, minimum 5 minutes. Allow the components to fully air dry before storage.
Maintenance	<p>Visually inspect to ensure that all contamination has been removed.</p> <p>If the shield is discolored, damaged, worn, or distorted it should be discarded.</p> <p>No additional maintenance or lubrication is recommended.</p>
Storage	<p>Store the sterilized light shield at room temperature, away from moisture or excessive humidity.</p> <p>Instruments steam autoclaved bagged should remain bagged until ready for use. Prior to subsequent re-use, inspect the sterilization pouch and the shield. If the integrity of the sterilization pouch has been compromised, the shield must be reprocessed prior to use. Instruments steam autoclaved unwrapped should be used immediately.</p> <p>Light handpiece and charging base should be cleaned, disinfected and dried as outlined in the section above prior to storage.</p>
Manufacturer Contact	Within the United States, call Dentsply Sirona at 1-302-422-4511. For areas outside the United States, contact your local Dentsply Sirona representative.

Incidental surface contact of the power supply and cord with water, soap or a water-based hospital-level disinfection solution will not damage the material of construction. Do not allow any solution to penetrate the casing.

5 Maintenance

5.1 Light output monitoring

1. Make sure that the LED aperture is clean and scratch-free; otherwise light output will be reduced and may be insufficient for proper curing of the material.
2. The light intensity of SmartLite® Focus® should be checked frequently to ensure appropriate curing by using a commercially available radiometer. As radiometers differ greatly, the following procedure is recommended.
3. Upon receipt of SmartLite® Focus® check the light intensity with the radiometer of your choice and record the value. This initial reading is for reference only, and may not represent the actual light intensity.
4. For subsequent monitoring retest the light intensity with the same radiometer that was used for the initial reference and compare it to the reference intensity.
5. If radiometer value drops below reference intensity, light efficiency can be verified using the i.Cure. Place i.Cure on a sheet of paper on a flat surface. Choose i.Cure segment according to step height required (please note that the step height should be twice the curing depth to be confirmed). Fill with composite. Hold SmartLite® Focus® in close proximity to the upper aperture and cure. If the material on the lower aperture has been cured (i.e. cannot be scraped off with a plastic spatula), the curing depth according to ISO 4049:2009 equals half the chosen step height. If i.Cure test is successful, SmartLite® Focus® is ready to use.
6. Do not continue to use SmartLite® Focus® if both light output is below reference intensity and i.Cure test has failed.

5.2 Battery 2

- Battery is equipped with low self discharge technology resulting in a long operating life.
- Battery is pre-charged and ready to use upon purchase. Charging before first use may improve initial operating time but is not mandatory.
- When the battery light flashes **green** 1.2, the battery is charging. Upon complete recharging, the battery light remains permanently green. The battery needs approximately 3 hours to be fully recharged. SmartLite® Focus® has a Smart Recharging Technology. Depending on battery status, SmartLite® Focus® will automatically choose between Automatic Quick Charging Mode (fast flashing green light) and Standard Charging Mode (flashing green light).
- The SmartLite® Focus® Automatic Quick Charging Mode allows a minimum of 5 minutes operating time after 10 minutes recharging.
- When the battery light flashes **yellow (fast)** 1.3, the battery needs to be recharged. At first occurrence approximately 1 minute curing time remain for completion of the treatment. Light output is not reduced during this period. After this period, the device switches off automatically and the battery light remains permanently **yellow** indicating the need to recharge.
- If the battery pack 2 needs to be replaced, release battery pack by gently inserting a screwdriver into the hole at the lower back end of the handpiece. The screwdriver is just needed to unlock the battery pack, please use fingers to completely remove battery pack as shown in **Figure A4** and **Figure A5**. Do not use the screwdriver as a lever, in order to avoid damage to the SmartLite® Focus® handpiece.

6 Reorder information, technical data, warranty terms

6.1 Accessories

Accessory	Reorder no.
SmartLite® Focus® battery pack 2	644.50.021
SmartLite® Sleeves 5	644.50.030
SmartLite® Focus® Shields 6	644.50.040
SmartLite® Focus® power connector incl. adapters 4	644.50.050
SmartLite® Focus® charging base 3	644.50.070
i.Cure	644.40.054

6.2 Serial number

Handpiece and battery pack have different serial numbers. Handpiece serial number can be found inside the handpiece after battery pack removal. Battery pack serial number is located on battery pack.

The serial number (**SN**) should be quoted in all correspondence which requires identification of the product.

Serial number format SmartLite® charging base:	SLXX	xxxxx
Serial number format SmartLite® Focus® handpiece:	SFXX	xxxxx
Serial number format SmartLite® Focus® battery pack:	SFBXX	xxxxx

6.3 Specifications and equipment class

Environmental Note

Device and battery disposal:

This device is provided with a nickel metal hydride battery. Device and battery must not be disposed in normal domestic waste. For environmental reasons, please dispose device and battery according to local environmental guidelines or regulations.

AC supply connection:	100 V...240 V/~ 50 Hz...60 Hz
Power input:	Maximum 14 W
Equipment class:	Class II equipment <input type="checkbox"/>
Follow Instructions for Use:	
Protection from ingress of liquids into handpiece:	IPX4
Operation:	Ambient temperature: Between 0 °C and +30 °C (32 °F and 86 °F) Relative humidity: Between 20% and 90%
Storage:	Ambient temperature: Between -10 °C and +35 °C (14 °F and 95 °F)
Transport:	Ambient temperature: Between -10 °C and +50 °C (14 °F and 122 °F)

Battery performance:	<ul style="list-style-type: none"> • Low self-discharge technology results in a long operating life. • Battery is pre-charged and ready to use upon purchase. • Time for battery recharge: Approximately 3 hours. • Automatic Quick Charging Mode: Allows a minimum of 5 minutes operating time after 10 minutes recharging.
----------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Battery over current/temperature protection:	Resettable fuse
----------------------------------------------	-----------------

Light emitting diode:	3 W LED
-----------------------	---------

Average light intensity:	Approximately 1000 mW/cm ²
--------------------------	---------------------------------------

Output peak wavelength range:	Between 460 nm and 490 nm (intensity maximum around 475 nm)
-------------------------------	----------------------------------------------------------------

Unit handpiece dimensions:	Length: 240 mm Width: 31 mm
----------------------------	--------------------------------

Unit weight:	Handpiece with battery pack: 90 grams Charging base with power connector: 200 grams
--------------	----------------------------------------------------------------------------------------

Complies with the following Directives/Standards:

93/42/EEC	Council Directive 93/42/EEC of 14 June 1993 concerning medical devices amended by directive 2007/47/EC, annex 1
2011/65/EU	Restriction of the use of hazardous substances in electrical and electronic equipment
IEC 60601-1	2012 – Medical Electrical Equipment (General requirements for basic safety and essential performance)
IEC 60601-1-2	2012 – Medical electrical equipment – Part 1-2: General requirements for safety and essential performance – Collateral standard: Electromagnetic compatibility – Requirements and tests
ISO 10650	2015 – Dentistry – Powered polymerization activators
IEC 62471	2006 – Photobiological Safety of Lamps and Lamp Systems

The SmartLite® Focus® curing light complies with:

6.4 Warranty terms

Dentsply Sirona grants a 2-year warranty on all components of the SmartLite® Focus® except the battery. The battery is covered by a 1-year warranty. The warranty commences on the date of purchase. Within the warranty period, Dentsply Sirona will eliminate free of charge any defects in the appliance resulting from faults in material or workmanship either by repairing or exchanging parts or exchanging the whole device at Dentsply Sirona's discretion.

Not covered by this warranty: Damage arising from improper use (operation with incorrect current/voltage, unsuitable power point, breakage, cleaning by other than the recommended methods), normal wear and defects which have a negligible effect on the value or operation of the appliance.

This warranty becomes void if repairs are undertaken by unauthorized persons.

This warranty extends to every country where this device is supplied by Dentsply Sirona or its appointed distributor and where no import restrictions or legal regulations hinder or prevent service being given under warranty.

Service under this warranty does not affect the expiration date of the warranty. The warranty on parts or entire devices which are exchanged ends when the warranty on this device expires.

In the event of a claim of this device, return the complete device (charging unit and the LED curing light) together with the invoice to your dealer or send it to your nearest Dentsply Sirona Service Center.

All other claims including those for damages resulting from this warranty are excluded unless our liability is legally mandatory.

7 Correspondence

1. The following numbers should be quoted in all correspondence:
 - Reorder number
 - Serial number
2. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.

© Dentsply Sirona 2020-10-05

[These Instructions for Use are based on Master Version 14]

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© Dentsply Sirona 2020-10-05

A Addendum SmartLite® Focus®

Guidance and manufacturer's declaration – Electromagnetic Emissions SmartLite® Focus®		
Declaration according to EN 60601-1-2: 2015 (ed.4)		
The SmartLite® Focus® is intended for the use in the electromagnetic environment specified below. The customer or the user of the SmartLite® Focus® should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The SmartLite® Focus® uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The SmartLite® Focus® is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes. The SmartLite® Focus® curing light is for use by trained dental professionals in dental offices or dental laboratories.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Guidance and manufacturer's declaration – Enclosure Port			
Table 4 according to EN 60601-1-2: 2015 (ed.4)			
The SmartLite® Focus® is intended for use in the electromagnetic environment specified below. The customer or the user of SmartLite® Focus® should assure that it is used in such an environment.			
Phenomenon	EN 60601-1-2 Immunity test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 20%.
RF electromagnetic fields IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz 80% AM for 1 kHz	3 V/m 80 MHz to 2.7 GHz 80% AM for 1 kHz	Refer to Table 5, Part B of Guidance and manufacturer's declaration
RF electromagnetic fields in the immediate vicinity of wireless communication devices IEC 61000-4-3	According to Table 9 of EN 60601-1-2	According to Table 9 of EN 60601-1-2	Refer to Table 5, Part B of Guidance and manufacturer's declaration
Magnetic fields with rated frequencies IEC 61000-4-8	30 A/m (50 or 60 Hz)	30 A/m (50 or 60 Hz)	

Guidance and manufacturer's declaration – Electromagnetic Immunity

Table 5, Part A according to EN 60601-1-2: 2015 (ed.4)

The SmartLite® Focus® is intended for use in the electromagnetic environment specified below. The customer or the user of SmartLite® Focus® should assure that it is used in such an environment.

Phenomenon	EN 60601-1-2 Immunity test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2 0.5 kV,	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 20%.
Electrical fast transient disturbances/ bursts IEC 61000-4-4	± 2 kV (100 kHz) for power supply lines ± 1 kV (100 kHz) for input/output lines	± 2 kV (100 kHz) for power supply lines ± 1 kV (100 kHz) for input/output lines	
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV line(s) to line(s) ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	± 0.5 kV, ± 1 kV line(s) to line(s) ± 0.5 kV, ± 1 kV, ± 2 kV line(s) to earth	
Voltage dips and variations on power supply input lines IEC 61000-4-11	0% U_T for 0.5 cycle (at 0, 45, 90, 135, 180, 225, 270, 315 degrees) 0% U_T for 1 cycle and 70% U_T for 25 cycles (50 Hz) or 30 cycles (60 Hz) single-phase: 0 degree	0% U_T for 0.5 cycle (at 0, 45, 90, 135, 180, 225, 270, 315 degrees) 0% U_T for 1 cycle and 70% U_T for 25 cycles (50 Hz) or 30 cycles (60 Hz) single-phase: 0 degree	
Short voltage interruptions on power supply input lines IEC 61000-4-11	0% U_T for 250 cycles (50 Hz) or 300 cycles (60 Hz)	0% U_T for 250 cycles (50 Hz) or 300 cycles (60 Hz)	

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – Electromagnetic Compatibility

Table 5, Part B according to EN 60601-1-2: 2015 (ed.4)

The SmartLite® Focus® is intended for use in the electromagnetic environment specified below.

The customer or the user of SmartLite® Focus® should assure that it is used in such an environment.

Phenomenon	EN 60601-1-2 Immunity test level	Compliance level	Electromagnetic environment – guidance
Electromagnetic fields radiated IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz 80% AM for 1 kHz	3 V/m 80 MHz to 2.7 GHz 80% AM for 1 kHz	<p>In order to prevent electromagnetic interference portable and mobile RF communications equipment should be used no closer to any part of the SmartLite® Focus®, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended minimum separation distance: $d = 1 \cdot \sqrt{P}$ for 150 kHz - 80 MHz $d = 2 \cdot \sqrt{P}$ for 80 MHz - 2.7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electro-magnetic site survey (a), should be less than the compliance level in each frequency range (b). Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Electromagnetic fields conducted disturbances RF IEC 61000-4-6	6 V 150 kHz to 80 MHz 80% AM for 1 kHz	6 V 150 kHz to 80 MHz 80% AM for 1 kHz	

Note 1: At 80 MHz the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a) Field strengths from fixed transmitters, such as base stations for radios (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the locations in which the SmartLite® Focus® is used exceeds the applicable RF compliance level above, the SmartLite® Focus® should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the SmartLite® Focus®.
- b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 6 V/m.

Manufactured by
Dentsply DeTrey GmbH
De-Trey-Str. 1
78467 Konstanz
GERMANY
www.dentsplysirona.com