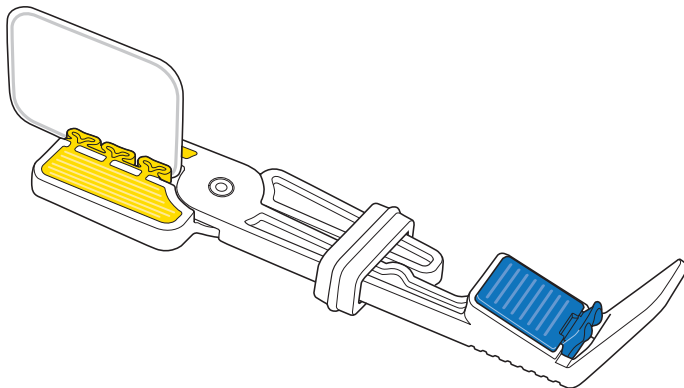
## Rinn<sup>®</sup> Snap-A-Ray<sup>®</sup> Xtra Film & Phosphor Plate Holder

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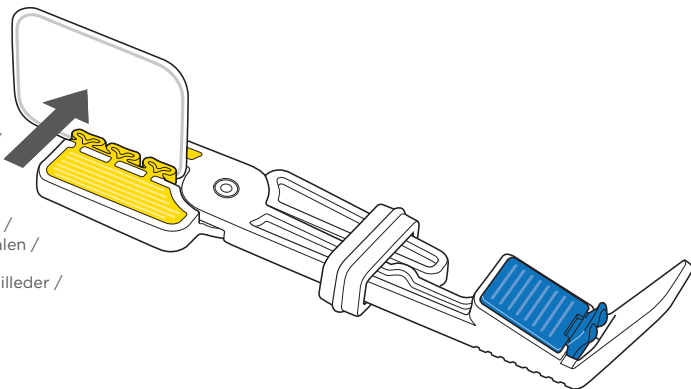
## Radiograph media can be angled for hard-to-reach areas.

Les supports de radiographie peuvent être inclinés pour les zones difficiles d'accès / Il supporto radiografico può essere angolato per le aree più difficili da raggiungere / Röntgenaufnahme-medien können für schwer zugängliche Bereiche angewinkelt werden / Los soportes para radiografía pueden ser angulados para áreas difíciles de alcanzar / Röntgenmedia kunnen onder een hoek worden geplaatst voor moeilijke bereikbare plaatsen / Nośnik radiograficzny może być nachylony w przypadku trudno dostępných obszarów / O suporte de raios X pode ser posicionado em vários ângulos para áreas de acesso difícil / Τα μέσα ακτινογραφικής απεικόνισης μπορούν να τεθούν υπό γωνία για δυσπρόσιτες περιοχές / Röntgenväline voidaan taivuttaa vaikeasti saavutettaviin alueisiin / Röntgenmedier kan vinkles til områder, der er svære nå / Röntgenmediet kan vinkles for områder som er vanskelig å bilde / Röntgenmedia kan vinklas för svåråtkomliga områden

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
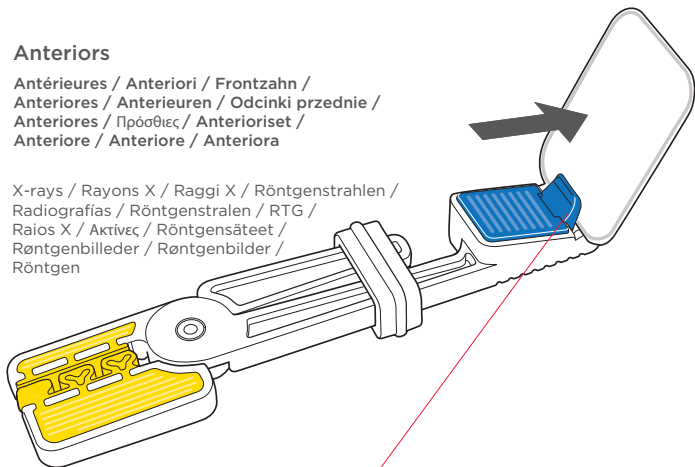
**To insert film or phosphor plate, pull blue media grip forward slightly.**

Pour insérer le film ou la plaque phosphore, tirer légèrement la poignée du support bleu vers l'avant. / Per inserire la pellicola o la piastra al fosforo, tirare leggermente in avanti l'impugnatura del supporto blu. / Zum Einführen des Films oder der Speicherfolie den blauen Mediengriff etwas nach vorne ziehen. / Para insertar la película o la placa de fósforo, tire ligeramente de la empuñadura azul del soporte. / U kunt de film of fosforplaat inbrengen door de blauwe mediahouder iets naar voren te trekken. / W celu umieszczenia folii lub płytki rentgenowskiej pociągnąć uchwyt niebieskiego nośnika delikatnie do przodu. / Para inserir película ou placas de fósforo, puxe ligeiramente o gancho do suporte azul. / Για την εισαγωγή πλάκας φωσφόρου ή φιλμ, τραβήξτε ελαφρώς μπροστά την μπλε λαβή του μέσου. / Asettaaksesi filmin tai fosforilevyn vedä sinistä välinekahvaa hieman eteenpäin. / Træk det blå mediegreb lidt fremad for at indsætte filmen eller fosforpladen. / Trekk det blå mediegrepet litt fremover for å sette inn film- eller fosforplaten. / För att sätta in film eller fosforplatta, dra det blå mediagreppet något framåt.

## In most cases, the x-ray cone should be aligned parallel to the blue bite surface.

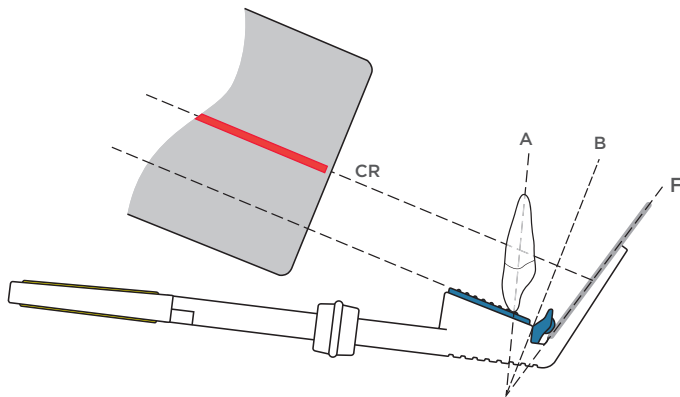Dans la plupart des cas, le cône de rayonnement doit être aligné parallèlement à la surface occlusale bleue. / Nella maggior parte dei casi, il cono di raggi X va allineato parallelamente alla superficie da mordere blu. / In den meisten Fällen sollte der Röntgenkegel parallel an der blauen Bissfläche ausgerichtet werden. / En la mayoría de los casos, el cono de rayos X debe estar alineado en paralelo a la superficie de mordida azul. / In de meeste gevallen moet de röntgenbundel parallel aan het blauwe bijtoppervlak worden uitgelijnd. / W większości przypadków wiązka stożkowa promieni RTG powinna być równoległa do niebieskiego wątku zwarciowego. / Na maioria dos casos, o cone de raios X deve estar alinhado de forma paralela com a superfície de mordida azul. / Στις περισσότερες περιπτώσεις, ο κώνος του ακτινογραφικού μέσου θα πρέπει να ευθυγραμμίζεται παράλληλα με την μπλε επιφάνεια δήξεως. / Useimmissa tapauksissa röntgenkeila on kohdistettava samansuuntaisesti sinisen purentapinnan kanssa. / I de fleste tilfælde bør strålebundtet vinkles parallelt med den blå bidoverflade. / I de fleste tilfeller skal røntgenkonusen innrettes parallelt med den blå biteoverflaten. / I de fleste fall skall røntgenkonen vara inriktad parallellt med den blå bitytan.

### BAI

#### Bisecting Angle Technique

Technique de la bissectrice / Tecnica dell'angolo di bisezione / Winkelhalbierende Technik / Técnica de ángulo bisectriz / Hoekdeellijntekniek / Technika dwusiecznych kąta / Técnica da bissectriz / Τεχνική της διχοτόμου / Kulmanpuolitus tekniikka (Bisecting Angle Technique, BAT) / Vinkelhalveringsteknik / Gjennomskjæringsvinkelteknikk / Bisekristeknik

### CR

#### Central X-ray

Rayon X central / Radiografia centrale / Vorderes Röntgen / Radiografias intraorales / Centrale röntgenbundel / Środkowa wiązka promieni RTG / Raios X central / Κεντρική ακτίνα / Keskitetty röntgen / Centralstråle / Midtstilt røntgen / Central röntgenstråle

### A

#### Longitudinal axis of the tooth

Axe longitudinal de la dent / Asse longitudinale del dente / Longitudinale Achse des Zahns / Eje longitudinal del diente / Longitudinale as van de tand / Wzdłużna oś zęba / Eixo longitudinal do dente / Επιμήκης άξονας του δοντιού / Hampaan pitkittäisakseli / Længdeakse af tanden / Tannens lengdeakse / Tandens longitudinella axel

### B

#### Imaginary bisecting line

Bissectrice imaginaire / Linea bisettrice immaginaria / Imaginäre winkelhalbierende Linie / Línea bisectriz imaginaria / Denkbeeldige hoekdeellijn / Wyobrazona linia dwusiecznych / Linha imaginária da bissectriz / Νοητή διχοτόμος / Kuvitteellinen puolituslinja / Imaginær halveringslinje / Imaginær gjennomskjæringslinje / Imaginär bisektrisinje

### F

#### Film plane

Plan du film / Piano della pellicola / Filmbene / Plano de película / Filmvlak / Płaszczyzna folii / Plano da película / Επιφάνεια του φιλμ / Filmitaso / Filmplan / Filmplan / Filmplan

## ENGLISH

**Indications for use:** Snap-A-Ray Xtra is indicated for holding film or PSP for intraoral dental radiographs.

**Contraindications:** None known.

**Warnings: Sterilize device before use.** DO NOT use phenol-based glutaraldehyde or ultrasonic cleaners. DO NOT chemically, dry-heat sterilize or cold sterilize. Only sterilize with steam sterilization according to the provided sterilization instructions. DO NOT exceed 273°F (134°C). It is the user's responsibility to qualify alternative methods of disinfection and sterilization. Failure to follow properly validated sterilization processes and approved aseptic techniques may result in cross contamination. Any method of sterilization will shorten the life of plastic parts.

**Precautions:** Follow sterilization instructions carefully to prevent damage to plastic pieces during sterilization.

**Adverse Reactions:** None known.

**Residual Risks:** Failure to follow the validated reprocessing instructions provided may result in exposing the patient to cytotoxic residuals and/or cross-contamination.

### Step-by-Step Instructions:

1. Insert film or PSP according to positioning guide.
2. Position holder in patient's mouth and align x-ray tube.
3. Take exposure.
4. Adjust as needed for the next desired position.

**Reprocessing:** Disassemble instrument(s) and separate all parts prior to reprocessing.

### Cleaning: Manual / Mechanical (Ultrasonic)

1. Rinse the disassembled device under potable running water while brushing with a soft-bristled brush to remove gross soil.
2. Prepare an enzymatic instrument cleaning solution and immerse the device per the manufacturer's instructions.
3. Rinse the device under potable running water for at least 30 seconds and dry using a disposable lint-free cloth.
4. Visually inspect the device for visible soil. If soil is seen, repeat the cleaning procedure.

### Cleaning and Disinfection: Automated – Instrument Washer-Disinfecter

Use only properly maintained, calibrated, and approved washer-disinfectors according to ISO 15883.

1. Rinse the disassembled device under potable running water while brushing with a soft-bristled brush to remove gross soil.
2. Follow manufacturer instructions for instrument washer solution, observing concentration rates and contact times as well as the instrument washer's recommended cleaning and/or thermal disinfection cycle (1 minute at 90°C).
3. Visually inspect the devices for visible soil and repeat the cleaning and disinfection procedure if necessary.

**Sterilization:** Ensure that the device is thoroughly dry prior to sterilization. Enclose the device into an appropriately sized sterilization pouch intended for steam sterilization. Place the pouched device into the sterilizer per the manufacturer's instructions. Sterilize the device according to one of the recommended methods listed below.

**Sterilize metal and plastic components in separate pouches to avoid damage to the components. DO NOT exceed 273°F (134°C).**

Steam Sterilization Cycle	Minimum Temperature	Minimum Full Cycle Exposure Time	Minimum Dry Time
Pre-vacuum	270 °F (132 °C)	3 min	16 min
Steam-Flush Pressure-Pulse	270 °F (132 °C)	3 min	16 min
Gravity Steam Sterilization	270 °F (132 °C)	10 min	15 min

**Alternate Method:** Place the non-pouched device into the sterilizer and sterilize according to one of methods listed above. Non-pouched device must be used immediately following sterilization.

**Storage:** To maintain sterility, the device should remain in the sterilization pouch until ready for use.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

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© 2019 Dentsply Sirona
MDSS GmbH