# Sockelformer

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REF 102640  
REF 102641  
REF 102642


# Base former

By means of the SILADENT base former, the model creation obtains a fast, economical and clean working basis. After 5 seconds only, the perfect base former is ready for each impression.


The simple handling and the high economic efficiency (low consumption of material) turns the base former into an important practice attendant.

## Working instruction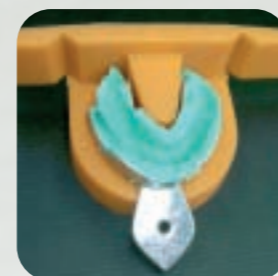


Check whether the impression is placed on a flat surface within the base former and remove unnecessary impression material from the tray.


Pour the gypsum into the impression mould as usually (please use a vibrator) and put it into the base former.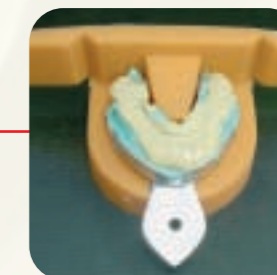

Close the base former arms around the impression tray softly with a rubber strap. Take care that the impression material does not get deformed.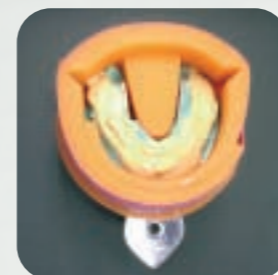

Afterwards fill up the base former with gypsum to reach a model basis with low material input.

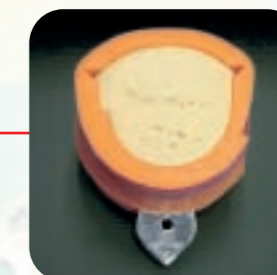If you do not use a gypsum trimmer, remove the base former before the gypsum gets too hard for cutting.

When the gypsum model has set, remove the impression from the mould and finalise the model as usually.

The base former is made from stable and tear resistant silicone. It's easy to clean and can be re-used many times.

**SILADENT Base former, set (upper & lower jaw)**  
SILADENT Base former (upper jaw)  
SILADENT Base former (lower jaw)

REF 102640  
REF 102641  
REF 102642