CE 0482

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Stand der Information: Februar 2017

CE 0482

## SPEIKOCAL Powder with x-ray contrast

### Composition:

Contains calcium hydroxide, barium sulfate.

### Fields of application:

- Supply of caries-free, near-the-pulp dentin
- Direct over-the-pulp capping, also after a pulpotomy
- Temporary insertion in the root canal
- Caries deep treatment
- For dental use only

### Contra-indications:

- In case of pulp opening in carious dentin
- In case of inflammation of changed pulp
- In case of hypersensitivity to ingredients

### Side effects/interactions:

None known if used according to intended purpose.

### Dosage, type and duration of treatment:

Mix the SPEIKOCAL Powder with distilled water until forming a paste. Add water to the mixing pad, add the powder with a spatula and process the mixture until obtaining a homogeneous paste. The consistency can be determined individually by adding water.

### Pulp capping (direct and indirect):

Apply the paste prepared from SPEIKOCAL Powder on the opened/exposed pulp or on the near-the-pulp dentin by using an instrument. After the application, the paste prepared from SPEIKOCAL Powder must be compacted by using water-absorbing materials (cotton pellets or paper points). Then it is coated tightly with a filling material.

### Temporary insertion in root canal:

Clean the root canal with 3% or 5.25% Hypochlorite-SPEIKO® and dry it with sterile paper points. The paste prepared from SPEIKOCAL Powder can be applied in the root canal as a temporary filling material for root canals with the help of a Lentulo spiral. If necessary, the application can be repeated. The calcium hydroxide paste must be removed prior to the definitive filling of the root.

### Caries deep treatment:

The cavity must be cleaned and dried prior to applying in a thin layer the paste prepared from the SPEIKOCAL Powder. If a thicker layer is desired, apply in several layers. Every layer must be dried briefly with a gentle flow of air. Then the underfilling and the filling are applied.

### Notes:

The SPEIKOCAL Powder may not be used after the expiration date.

Storage at room temperature.

### Trading format:

SPEIKOCAL Powder                      Content: 15 g                      Art. no. 1050

Accessories: Dosage spoon for powder

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