## Super-Splint/ Super-Splint De Luxe

D

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## Super-Splint/ Super-Splint De Luxe

GB

**Super-Splint (De Luxe) is a multiple layer fiberglass strip for strengthening dental resins.**

**It is used for the following indications:**

- Splinting loose anterior teeth caused by periodontitis or trauma.
- Re-implantation of extracted or totally luxated anterior teeth to act as a long-term temporary bridge.
- Reinforcing crowns and bridges fabricated in the mouth or laboratory.
- Reinforcing acrylic denture repairs.

### INSTRUCTIONS FOR USE

1. Place a rubber dam.
2. Clean all involved teeth with non-fluoride polishing paste.
3. Etch the oral and interdental surfaces of the teeth for 60 seconds then rinse and blow dry.
4. If necessary the teeth can be fixed temporarily by placing a small drop of bonding agent in each of the interdental spaces.
5. Coat the oral and interdental surface with a bonding agent.
6. Cut a piece of Super-Splint (De Luxe) defining the appropriate length, by using a perioprobe.
7. Saturate it with the bonding agent on glass slab.
8. Place Super-Splint (De Luxe) strip onto the oral surface and hold it in place with the application clips.
9. The strip is kept in position while polymerized 30 – 40 seconds per interproximal space.
10. Any rough areas can be smoothed with a layer of flowable composite. The splint is then trimmed and polished. Remove the rubber dam and check the occlusion.

**Notice: The product is for single use only. In case of a re-use its functionality can no longer be guaranteed and will highly increase the danger of infection.**

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