Temdent Classic

SCHÜTZ  **DENTAL**
Micerium Group

Schütz Dental GmbH • Dieselstr. 5-6 • 61191 Rosbach/Germany
Tel.: +49 (0) 6003 814-0 • Fax: +49 (0) 6003 814-906
www.schuetz-dental.de • info@schuetz-dental.de

CE 0297

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Direction for use

Temdent Classic is a powder / liquid system for fabricating temporary crowns and bridges. Temdent Classic is a cold-curing polymer with a poly-methyl methacrylate base that contains just 0.8 % residual monomer.

Temdent Classic is suitable for both the direct and indirect methods.

Mixing and Adjusting the Colour

Mix Temdent Classic in a cup. You can use any dosage.

The viscosity should be creamy or like dough. This corresponds to about 1.8-2.2 g powder to 1 g liquid (mixing ratio is about 2:1).

Fill the cup with the liquid. For an average crown, you will need about 8 drops (0.15 g) and around 0.3 g of powder. You can adjust the colour by mixing light and dark powders together.

Indirect Method

Insulate situ model with silicone spray. Cover with a deep-drawing foil with help of a deep-drawing device. Trim off excess material. To stabilize thin edges, we recommend the use of a commercially available deep-drawing foil. Mix Temdent Classic as described and directly pour it into the matrix. The time period from adding the powder to the liquid to the application of the product must not exceed 2 minutes. The matrix foil must be reduced exactly to the model to avoid a raising of the bite and to prevent later corrections inside the mouth.

After about 6 minutes, carefully lift the temporary appliance with matrix off the model. Now the elastic phase of about 1½ minutes begins. In this time, overlapping margins can be removed with an instrument. The temporary appliance remains on the model for final curing (about 2 minutes). After final curing, grind and polish the margins. Temdent Classic is repairable.

Direct Method

Impression Technique

Before preparing the tooth, take an alginate or silicone impression (e. g. Trealgin, Tresident, Schütz Dental Group) and store it in the usual way.

After preparation, mix Temdent Classic as described above.

Remove interdental septa if necessary and blow the impression dry. Pour the mixed Temdent Classic into

the desired area of the impression. It takes about two minutes for the material to set. Before applying the impression filled with Temdent Classic in the mouth, you need to ensure that the prepared stumps are insulated (e. g. with Vaseline).

Now apply Temdent Classic with the impression in the mouth.

Temdent sets slowly. After about 3½ minutes, you can remove the temporary appliance in the rubber-elastic phase. This step prevents Temdent Classic being suddenly bonded to prepared teeth. If the temporary appliance adheres to the preparation, it must be carefully removed and put back in the impression for curing. After curing, the temporary appliance is dimensionally stable and can now be trimmed.

Matrix Technique

As already described, a foil that is 0.6-0.8 mm thick is deep-drawn over a diagnostic model insulated with silicone spray.

Mix Temdent Classic as described above and cast it in the matrix. The time from when the powder is added to the liquid must not exceed 2 minutes.

The matrix foil must be repositioned exactly on the teeth. The patient's mouth is closed and later grinding in the mouth is unnecessary. After about 3½ minutes, carefully remove the temporary appliance with matrix from the mouth. Further processing is carried out as described above.

Storage

The material should be stored at 10-25 °C. Seal the container securely after use. Keep away from ignition sources and do not smoke.

Disposal

The powder can be disposed of in domestic refuse. The liquid must be disposed of in hazardous waste.

Guarantees

Our processing instructions, regardless of whether they are issued verbally, in writing or in the form of a practical demonstration, are based upon our own experience and can therefore be regarded solely as guidelines.

Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (e.g. allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

Contra-indications/interactions

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under

strict medical supervision. In such cases, we will supply the composition of our medical device upon request. The dentist should consider known interactions and cross-reactions of the product with other materials already in the patient's mouth before using the product.

Note

Please pass all the above information to the dentist carrying out the treatment if you are using this medical device for a custom-made product. Please observe all existing safety data sheets during processing.

Troubleshooting

Problem	Cause	Solution
Plastic is not curing properly	Wrong mixing ratio	– Follow the directions for use – Use the correct mixing ratio
Plastic is curing too quickly / too slowly	Unfavourable ambient temperature High temperatures accelerate polymerisation, while low temperatures slow it down	Adjust the procedure
Colour is not right	Light and dark powders can be mixed individually	Select another powder mixing ratio.
Temporary appliance is adhering to the prepared tooth	It has been left too long in the mouth	Follow the directions for use

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