


UPOFIX® - POLISHERS

Brush	Application	Field of application
0	rough polishing	all metals
I	prepolishing	all metals
II	final polishing	all metals
III	prepolishing	ceramics
IV	final polishing	ceramics
IV	pre polishing	composite
V	final polishing	composite

Attention: 3' – 5.000 rpm (slow with increased pressure)

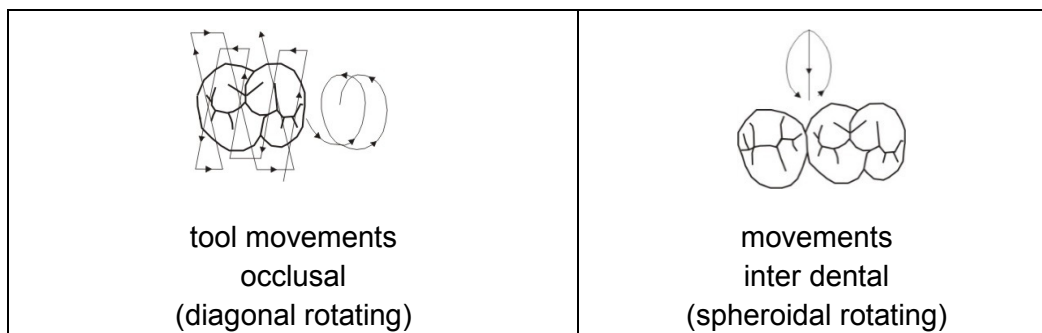

Low rpm – 3000 rpm and increased surface pressure: abrasive (pre-)polishing

Increased rpm - 5000 rpm and low up to no surface pressure: final polishing

For more information see catalogue Dental Lab

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 12/14