**UltraTemp®**  
Temporary Cement  
Regular  
Firm-Regular Set  
Firm-Fast Set

**Description**

**UltraTemp®** and **UltraTemp® Firm** are non-eugenol temporary cements. They are polycarboxylate based and do not negatively affect resin bonding. Being water soluble until set, they clean up easily. UltraTemp Firm is available in regular and fast set.

**Indications for Use**

For temporary application of provisional crowns, bridges, inlays, and onlays.

**Directions for Use**

1. Remove cap by twisting counterclockwise and discard.
2. Prior to attaching mixing tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate orifices.
3. Attach mixing tip by lining up internal cap stems with syringe orifices. Twist clockwise to lock.
4. Clean, rinse, and dry preparation.
5. If using intraorally, firmly attach an Intraoral tip.
6. Express desired amount of UltraTemp® into provisional or access opening.
7. Seat provisional. Remove excess using a wet cotton pellet.
8. After complete set, remove all excess cement.

|                              | Set Time      |
|------------------------------|---------------|
| UltraTemp                    | 4 - 5 minutes |
| UltraTemp Firm - Regular set | 2 - 3 minutes |
| UltraTemp Firm - Fast set    | 1 - 2 minutes |

9. Disinfect and wipe mixing tip and syringe. Do not remove mixing tip until next use.

## **Removal of UltraTemp® from provisional and tooth surface**

1. Remove provisional.
2. Flake off residual cement with blunt hand instrument.
3. Scour preparation with pumice-type slurry and cup or brush.
4. Rinse thoroughly prior to placing definitive restoration.

## **Precautions and Warnings**

1. Carefully read and understand all instructions before using.
2. Do not use on patients with known sensitivities to polyacrylic acid (polycarboxylate).
3. Avoid cross-contamination of dual chambers which could result from repeatedly attaching and re-attaching the same mixing tip.
4. Dispose of used tips and empty syringe properly.
5. Verify flow prior to applying intraorally. If resistance is met, replace tip and re-check.
6. Do not refrigerate.
7. Keep UltraTemp® away from eyes.
8. When using UltraTemp®, do not remove mixing tip until next use. Disinfect and wipe syringe and mixing tip with an intermediate level disinfectant between uses. Change mixing tip before each use.
9. Patient should be anesthetized to prevent discomfort during cementation with UltraTemp® or any polycarboxylate or acid-based cement.

**NOTE: For MSDS and additional information about using UltraTemp®, please go to [www.ultradent.com](http://www.ultradent.com)**

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

Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

**UltraTemp Firm ( Base & Catalyst )**

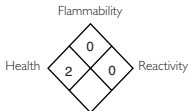**For product MSDS please see our website: [www.ultradent.com](http://www.ultradent.com)**

**UltraTemp ( Base & Catalyst )**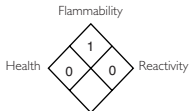

**For product MSDS please see our website: [www.ultradent.com](http://www.ultradent.com)**

HAZARD RATING  
4 = Severe  
3 = Serious  
2 = Moderate  
1 = Slight  
0 = Minimal

Federal law restricts this device to sale by or on the order of a dentist.  
Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.  
Outside U.S. call (801) 572-4200  
or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2007 Ultradent Products, Inc. All Rights Reserved.