Ultra-Etch® 35% Phosphoric Acid

Description:

UltraEtch is a 35% phosphoric etchant solution with an optimum viscosity. The flowability allows precise placement, including occlusal grooves, yet is viscous enough to prevent migration. UltraEtch is self-limiting in its depth of etch (average depth of 1.9µ with 20 second etch). This etchant contains no glycerin and is designed for rapid and complete removal upon rinsing.

Indications for Use:

Used for etching enamel and dentin prior to applying bonding adhesives and placing composite restorations or dental sealants.

Directions for Use:

1.For 1.2 syringe

a.Remove Luer cap.

b.Securely attach working tip of choice (Blue Micro® or Inspirai Brush® tip).

c.Verify etchant flow prior to applying intraorally.
d.Verify the color and consistency of the product flow before applying. Initially a small amount of clear liquid may express from the syringe; this is normal.

2.For IndiSpense Syringe (Figure 1 & 2)

a.Remove Luer cap from Indispense syringe.

b.Attach 1.2 syringe to the male threads of the IndiSpense syringe.

c.Depress IndiSpense plunger while guiding 1.2ml syringe plunger to desired fill.

d.Separate syringes and re-cap IndiSpense syringe with Luer cap.

e.Securely attach working tip (Blue Micro® or Inspirai Brush® tip) to 1.2ml syringe.

3.Direct placement instructions

a.Isolate tooth (teeth).

b.Use pumice, disk or diamond bur on uncut enamel.

c.Rinse and dry prepared area.

d.Verify etchant flow prior to applying intraorally.
e.Apply etchant to enamel and dentin (20 seconds)

f.Rinse thoroughly, dry and proceed per adhesive manufacturer’s instructions.

4.Restorative phosphoric acid cleansing

a.Apply etch to inside of porcelain veneer or crown for 5-10 seconds.

b.Rinse and dry

Precautions and Warnings:

1.Carefully read and understand all instructions before using.

2.Keep out of reach of children.

3.If product comes in contact with any soft tissue, IMMEDIATELY rinse area with copious amounts of water.

4.Near pulp exposures should be treated with a protective base prior to placing etchant.

5.Re-cap, disinfect and wipe syringe with an intermediate level disinfectant between uses. If syringe cover is used, remove tip, re-cap, and discard syringe cover

6.Dispose of tip and empty syringe properly.

7.Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips. To avoid cross contamination, do not re-use tips.

8.Store at room temperature.

9.Do not use after expiration date noted on container.

NOTE: For SDS and additional information about using Ultra-Etch, please go to www.ultradent.com

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