# Rinn® Uni-Grip® 360

## Universal Sensor Holder

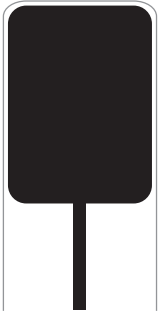
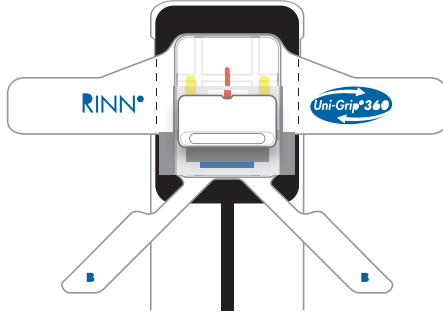
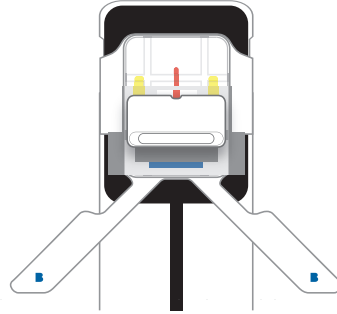
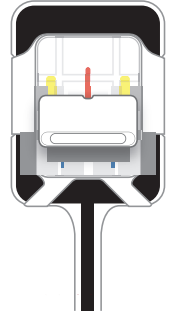
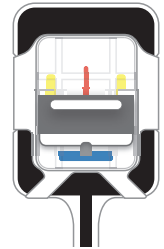
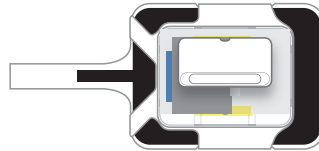
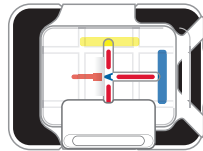
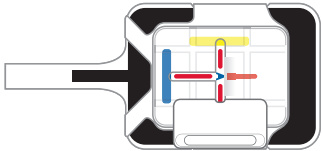
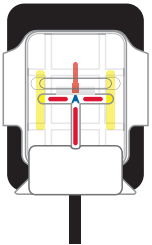
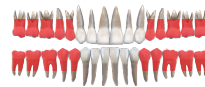
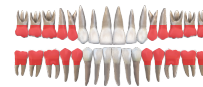
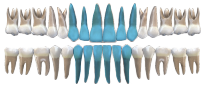
Porte-capteur universel / Porta-sensore universale / Universal-Sensorhalter / Soporte de sensor universal / Universele sensorhouder / Uniwersalny uchwył czujnika / Suporte de sensor universal / Βάση αισθητήρα γενικής χρήσης / Anturin yleispidike / Universal sensorholder / Universal sensorholder / Universalsensorhållare

REF
-----

ReOrder No.

550052	Uni-Grip® 360, Package of 50 with Sensor Covers
550252	Uni-Grip® 360, Package of 100 with Sensor Covers (Includes Uni-GripAR® Positioning Arm and Ring)
550100	Uni-GripAR® Positioning Arm and Ring
550500	Rinn Universal Sensor Cover, Package of 500

**A****B****C****D****E**

## Step-By-Step Instructions

1. Cover digital sensor with sensor cover included in the Uni-Grip® 360 package. **A**
2. Remove backing liner from Uni-Grip® 360 and apply to center of active area with adhesive tabs marked **B** towards the bottom of the sensor where cord exits. **B**
3. Tightly wrap side tabs first **C** and then bottom tabs to back of sensor. **D**
4. Slide bite tab to desired position **E** and take x-ray. Repeat as necessary.
5. To remove, lift base to separate perforations on side tabs **F** and then remove bottom tabs. Do not pull on sensor cord when removing sensor from sensor cover.

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## Usage Tips

1. The Uni-Grip® 360 bitetab can rotate anywhere on the base. If the bitetab becomes separated from the base, it can be reinserted at point **A. G**
2. The bitetab can extend past the edge of the base for a wide range of positioning options. To capture more area past the apices, make sure tab is fully extended past base **H**. To capture more of the incisal edge, the bitetab can be adjusted up on base. **I**
3. For maximum patient comfort, always place sensor in the center of mouth.
4. Instruct the patient to "bite lightly" to hold sensor in place.
5. For additional assistance in the United States, call RINN Customer Service at 800-989-8826.

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**Indications for Use:** Rinn Uni-Grip Sensor Holder is indicated for use as a digital sensor holder during intraoral radiographs.

**Contraindications:** None known.

**Warnings:** The Uni-Grip Holder is single use only. Do not reuse.

**Precautions:** To be used only by individuals professionally trained to take intraoral radiographs. During use wear protective gloves. When using with digital sensor, only apply to protective sensor cover. Do not pull on sensor cord when removing. Only apply to dry surfaces.

**Adverse Reactions:** None known.

**Residual Risks:** Reuse of single use devices can result in cross-contamination and lead to patient infection or illness.

**Storage:** Store at room temperature (65-75°F/18-24°C), out of direct sunlight.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

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MDSS GmbH