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**INDICATIONS**

Mixing liquid for mixing VINTAGE Body and Enamel porcelains. It is designed to keep moisture longer than distilled water.

**INGREDIENTS**

Distilled water, 1,3-Butanediol

**NOTE ON USE**

- Use VINTAGE Mixing Liquid instead of distilled water or mix with distilled water to obtain desired handling characteristics.
- If additional mixing is required to obtain desired handling, add distilled water.
- Do not use with Opaque porcelain.

**CAUTION**

- Do not use or keep near fire as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperature or direct sunlight.
- Keep lid tightly closed if not in use.
- Do not use for purpose other than that described here.
- This product is for use by dental professionals only.

**IMPORTANT**

- Do not touch with hand. Pay particular attention to avoid contact with skin or eyes.
- In case of contact with skin, wipe off immediately with cotton wool soaked in alcohol and rinse with running water.
- In case of contact with eye, rinse immediately with plenty of water and seek medical advice.
- If skin eruption or inflammation occurs, stop using the product and seek medical advice.

**STORAGE**

- Keep away from heat, direct sunlight or flame. Seal the cap tightly and keep at room temperature.
- This product must be kept away from all personnel except dental professionals.

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