Rinn® XCP®

## Endodontic Instrument


Instrument endodontique / Strumento endodontico /  
Endodontisches Instrument / Instrumento de endodoncia  
/ Endodontisch instrument / Narzędzia endodontyczne /  
Instrumento endodôntico / Ενδοδοντικό εργαλείο / Juurihoidon  
instrumentti / Endodontisk instrument / Endodontisk  
instrument / Endodontiskt instrument


### Maxillary

Maxillaire / Mascellare /  
Oberkiefer / Maxilar / In  
bovenkaak / Szczeka / Maxilar  
/ Άνω γνάθου / Maksillaarinen  
/ Overkæbe / Overkjeve /  
Överkåken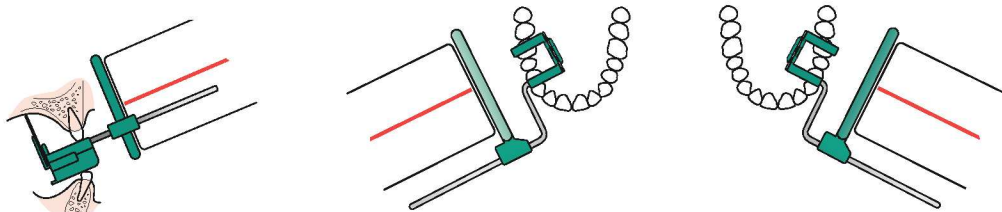

### Mandibular

Mandibulaire / Mandibolare  
/ Unterkiefer / Mandibular  
/ In onderkaak / Zuchwa /  
Mandibular / Κάτω γνάθου /  
Mandibulaarinen / Underkæbe /  
Underkjeve / Underkåken

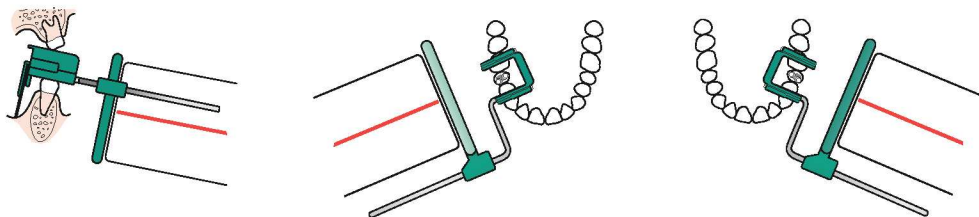## Positioning Instructions

1. Assemble instrument. Be sure the biteblock is centered in aiming ring.
2. Place appropriate size film/Phosphor plate in instrument with imprinted side of film away from x-ray tubehead.
  - "White toward light" — place plain or white side of film toward x-ray tubehead.
3. Position instrument in patient's mouth as posterior as possible, away from the teeth and in the mid-palatal or mid-tongual area. Center film/Phosphor plate on the teeth to be radiographed (see diagrams).
  - "Lean and tip" — lean the chair back just a little and/or tip the patient's chin up to facilitate placement of the XCP instrument into the center of the patient's mouth. For exposure, the occlusal surface should be parallel to the ground.
  - For endo, place the instrument over the tooth, files and clamps, resting on the adjacent teeth. If using a rubber dam, release the top or bottom of the dam from the frame. The dam and frame do not need to be removed.
4. Instruct patient to close slowly and bite. A cotton roll can be placed between the biteblock and the opposite teeth for added comfort and stability.
  - "Gentle; breathe" — ask the patient to gently bite down and then take several breaths; it gives them something to concentrate on other than the radiograph procedure.
5. Position tubehead parallel to indicator arm and centered on aiming ring.
6. Make exposure.

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© 2019 Dentsply Sirona
MDSS GmbH