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ZAKK®
CE 0482

Hersteller

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 040 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

ZAKK®

REF ZKS2070

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ZAKK®
REF ZKS2070

Description and indication

ZAKK® in a 1:1-double-chamber-syringe (KwikkMix®-System) is used as an universal, non-eugenol temporary crown and bridge ZnO-cement for cementing temporary crown and bridges and for trial cementing definitive crown and bridges.

ZAKK® blends with aesthetic neutral, universal shade for all commercial temporary restorations.

ZAKK® cures fast for an optimal adaptation and guarantees at the same time an easy removal of excess material.

ZAKK® shows a reversible character, that is an excellent adhesion without any loss of retentions to temporary restorations and at the same time an excellent removability from prepared teeth.

ZAKK® is based on zincoxide and organic acids and contains dimethacrylates for an optimal marginal seal without wash-out effects.

The non-eugenol ZAKK® will not influence hardening of self-cure - or dual-cure resin-based composites or luting materials for definitive restorations.

Especially in cases of allergies to eugenol ZAKK® is strongly recommended to use.

ZAKK® meets the requirements of DIN EN ISO 3107 and the ADA/ANSI-Specification No. 30.

Recommended procedures**1. Preparation of 5 ml-double-chamber syringe (KwikkMix®)**

Before first use screw off the original cap and waste it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally. Then the mixing cannula is put on and locked by turning it 90° clockwise.

ZAKK® will be mixed automatically using the MIXING CANNULA MCN 2 (REF MIK1041) + INTRA-ORAL-TIP MCN 2 (REF IOT2041) of the safety syringe system.

Instead of the INTRA-ORAL-TIP MCN 2 a fine root-canal application tip is available (ROOT-CANAL-TIP, REF RCT2041), with which the material could be applied directly and efficiently.

Further on instead of of MIXING CANNULA MCN 2 + INTRA-ORAL-TIP MCN 2 a NEEDLE MIXING CANNULA (REF MIK1041N) with an integrated, bendable application needle is available.

For this type of double-chamber syringe a mixing pistol for a comfortable application is available (REF MIP1041).

2. Application

Recommendation: Do not dry the stumps totally. A slight humid tooth surface will allow to remove the temporary restoration more easily.

Dispose first a small amount of ZAKK® after extruding. Then introduce the material into the temporary restoration and put it onto the prepared areas.

ZAKK® should be introduced within 90 s into the mouth (working time). The intra-oral polymerization time is about 2-3 min. Then any excess material can be removed easily with a scaler.

After removal of the temporary restoration the polymerized cement can be easily separated from the tooth.

Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

Precautions to application, product life and/or storage

Avoid contact of ZAKK® with soft tissues, skin or eyes. If there is an accidental contact, flush immediately with numerous amounts of water and contact a physician if necessary.

With susceptible patients, sensitization to ZAKK® cannot be excluded. Do not use ZAKK® for patients with allergic reactions against methacrylates. If swallowed accidentally, immediately consult a physician.

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Before first use screw off the original cap and dispose it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally.

Before application a small amount of material (size like rice corn) should be pressed out and should be disposed of immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap.

Shelf life see label. Do not use after expiry date.

Do not store above 78 °F!

Avoid exposure to sunlight. Keep out of the reach of children!

The double-chamber syringe should be stored horizontally.

Technical data

Mixing time: auto-mixing

Working time incl. mixing time: < 90 s
Intra-oral polymerization time (self-curing): about 2-3 min

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2019 R-dental Dentalerzeugnisse GmbH. All rights reserved.