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ZAKK® Beauty

CE 0482

Hersteller

R-dental Dentalerzeugnisse GmbH - Hamburg
Telefon 0 40 - 22 75 76 17 - Fax 0 40 - 22 75 76 18 - www.r-dental.com

ZAKK® Beauty**Bestell-Nr.: ZKS2075**

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ZAKK® Beauty**Order-No.: ZKS2075**

ZAKK® Beauty in a 4:1-double-chamber-syringe (KwikkMix®) is an automixing, non-eugenol temporary crown and bridge cement on methacrylate basis for cementing temporary crown and bridges.

ZAKK® Beauty can be light-cured and is self-curing at the same time (dual-cure).

ZAKK® Beauty blends with an excellent esthetics translucent appearance, compatible to all high class temporary restorations.

Because of its translucent appearance ZAKK® Beauty can be used excellently as a temporary Try-In- and test cement for definite restoration (like crowns and bridges).

ZAKK® Beauty shows an excellent adhesion and removability without any loss of retentions towards prepared teeth or restoration. It leads to an optimal adaptation and at the same time an easy removal of excess material at the margin.

ZAKK® Beauty is based on methacrylate with zinc oxide/calcium oxide and it polymerises to an elastic cement. The relative intraoral insolubility guarantees an optimal marginal seal without wash-out effects.

The non-eugenol ZAKK® Beauty will not influence hardening of definitive placing light-cure -, self-curing - or dual-cure resin-based composites or luting materials.

Recommended procedures**1. Preparing the double chamber syringe**

First remove the original cap of the double chamber syringe and screw on the MIXING CANNULA MPN 2 (Order-No.: MIK1046). The double chamber syringe is ready for application.

2. Application

Dispose first a small amount of ZAKK® Beauty after extruding. Then introduce the material with help of the mixing cannula into the temporary restoration and put it onto the prepared areas (Recommendation: Do not dry the stumps totally. A slight humid tooth surface will allow to remove the temporary restoration more easily).

The ZAKK should be introduced within 2 min 30 s into the mouth (working time). The intraoral hardening time is more than 4 min (setting time about 6 min 30 s). Then any excess material can be removed easily with a scaler.

After removal the polymerised cement can be easily separated from the tooth and from the provisional or restoration.

Times mentioned for 74 °F. Higher temperatures shortens, lower temperatures prolongs these times.

Special storage and/or handling conditions

The filled MIXING CANNULA MPN 2 functions as a seal until the next application. Do not re-use the cartridge cap.

The shelf life is at least 2 years, when stored at 74 °F. Do not store above 77 °F!

Technical data

Mixing time:	auto-mixing
Working time incl. mixing time:	< 2 min 30 s
Light polymerization time (light-cure):	~ 20 s
Hardening time in mouth (self-curing):	> 4 min

ZAKK® and KwikkMix® are registered trademarks of R-dental Dental-erzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

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Material Safety Data Sheet according to Regulation (EC) No. 1907/2006**1. Product identification/preparation/company undertaking**Trade name: **ZAKK® Beauty**Indication: **Esthetic temporary crown - and bridge cement**Order-No.: **ZKS2075**

Manufacturer: R-dental Dentalerzeugnisse GmbH, Germany.

Emergency information:

Dr. A. Sprafke, Phone: +49 (0) 40-22 75 76 17

E-mail: info@r-dental.com

Web: www.r-dental.com

2. Hazards identification

Risk identification: -

Classification: Classification corresponds to present EEC guidelines, additionally information by literature.

3. Composition/information of ingredients

Chemical characterization: Mixture of methacrylates, unsaturated esters, Zincoxide/Calciumoxide, Si-Oxide, pigments, stabilizers, initiators.

Description: -

Hazardous components: -

CAS-Nr.: [1314-13-2]

Name: Zincoxide

R-Classification: N, R50/53

CAS-Nr.: [607-134-00-4]

Name: METHACRYLIC ACID ESTER

R-Classification: Xi, R 36/37/38

4. First-aid measuresGeneral information:

General information after inhalation: Take fresh air.

General information after skin contact: Wash with plenty of water and soap.

General information after eye contact: Rinse with plenty of water and consult a physician.

General information after swallowing: Seek medical advice immediately.

5. Fire fighting measuresExtinguishing media: CO₂, water.

Protective equipment: Working clothes.

6. Accidental release measures

Personal precautions: Working clothes.

Environmental precautions: Do not let material come into contact with the environment.

Methods for cleaning up not cured material: Pick up the material.

Methods for cleaning up cured material: Pick up the materials.

Forward for disposal.

7. Handling and storage

Handling procedures: This product should only be used by a dentist, dental technician or a licensed professional.

Storage: Store horizontally at dry place, at room temperature or cool and away from food stuffs.

Requirements for storage with other products: -

Additional storage conditions: -

Storage classification: -

VBF-classification: -

8. Exposure controls/personal protection

Additional information: The product contains no relevant quantities with limits of values to be supervised at the workplace.

Personal protective equipment: -

General measures of protection and hygiene: Avoid contact with eyes or skin.

Respiration: -

Hands: Wear protective gloves.

Eyes: Wear eye glasses.

Pregnancy: -

9. Physical and chemical propertiesAppearance:

Form: Paste.

Colour: Tooth shades.

Odour: Relative odourless.

Information on change in the physical state (Value, unit, method):

Melting point/melting range: -
Boiling point/boiling range: -
Flash point: -
Autoignition temperature: -
Danger of explosion: -
Density: 1,56 g/cm³ (20 °C, 68 ° F).
Vapour pressure: -
Viscosity: Flowable.
pH: -
Solubility in/miscibility with: -
Water solubility: Not soluble.
Content of solvents: -
Organic solvents: -
Water: -
Content of solids: -

10. Stability and reactivity

Stability and reactivity: Avoid heat and sunlight.
Incompatibility with other substances: Polymerizes in contact with peroxides, metal powders, strong oxidizers and free radical starters.
Dangerous reactive substances: -

11. Toxicological information

Allergic reactions to methacrylates by chronic exposure to methacrylates are possible.
Acute toxicity: -
Primary irritation skin: Slightly irritating.
Eye: Irritating.
Inhalation: -
Ingestion: -
Sensitization of product: Not observed.
Additional toxicological information: During the long period of use as methacrylate based materials no health diseases are reported.

12. Ecological information

General information:

Degrance: Do not let the not cured product come into contact with the environment.
Fish toxicity: Do not let the not cured product come into contact with the environment.
Classification of water endangerment, WGK: -
Additional information: Do not let the not cured product come into contact with the environment.

13. Disposal consideration

Recommendation: Use proper landfill disposal or incineration in accordance with local, state and federal regulations.
Disposal key number: -
Uncleaned packaging: -

14. Transport information

Transport information: Not classified as dangerous goods.
Land transport RID/ADR, GGVS/GGVE: Not classified as dangerous goods.
Ship transport ADN/ADNR: Not classified as dangerous goods.
Sea transport IMDG/GGVSea: Not classified as dangerous goods.
Air transport ICAO-TI and IATA-DGR: Not classified as dangerous goods.

15. Regulatory information

Classification according to EEC-guidelines (according to GEFStoffV/EG):

N Dangerous for the environment
Xi Irritating
R 36/37/38 Irritating eyes, respiratory tract and skin
R 50/53 Very toxic to aquatic organisms, may cause long term adverse effects in the aquatic environment

National prescriptions: -

16. Other informations

The above information is based on our present day knowledge and relates solely to the safety requirements of the product. The data do not signify any warranty with regards to product properties. However users of the product should satisfy themselves that the information given above is sufficient and correct for their circumstances of use.

Prepared by: Dr. Andreas Sprafke, R-dental GmbH.

Version: 09.2009