AC Applicator tipo 2

MD UE Dispositivo médico

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MD EU Medical device

Instructions for use

Direct application dispenser for glass ionomer restorative materials and luting cements in application capsules (AC).

The parallel lever movement of the applicator allows a simple and precisely dosed application of the material. The small aperture of the application cannulae allows the material to be applied into the cavity or the restoration for attachment with precision.

Patient target group:

AC Applicator type 2 is suitable for use in all patients without any age or gender restrictions.

Performance features:


The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:


AC Applicator type 2 should only be used by a professionally trained dental practitioner.

Application:



1. Pull the plunger backwards to the stop.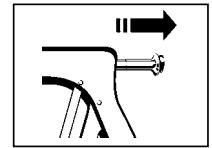

2. Insert the unopened application capsule into the applicator following activation and completion of mixing. Then carefully push the plunger slightly forward until it touches the rear wall of the capsule.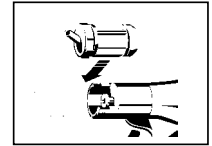

3. Open the inserted application capsule's twist cap with application cannulae by turning it through 180° (see imprint on the cap) until you feel it hit the stop. Rotate the application capsule in the desired direction and apply its content into the cavity by slowly and evenly compressing the lever. Do not apply excessive pressure.

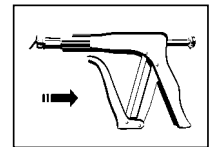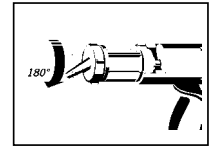4. On completing the application, pull the plunger back again and lift the application capsule out.

5. Disinfect the applicator after each use. Conventional disinfectant wipes or solutions (except acetone) can be used for disinfection. The **AC Applicator type 2** can be steam sterilised up to a maximum of 134 °C (steam steriliser, follow manufacturer's instructions).

Warnings, precautionary measures:

- The **AC Applicator** application system enables you to quickly and hygienically apply the material directly.
- For detailed information on the different **VOCO** products, please consult the individual instructions for use.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **AC Applicator type 2** must be reported to VOCO GmbH and the responsible authority.

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