VOCO**Bifix® QM**

Instructions for use

**Product description:**

Bifix QM is a radiopaque, dual-curing composite-based luting system for permanent adhesive luting of metal, ceramic and composite restorative pieces.

Bifix QM is available in three shades: universal, transparent and white-opaque. QuickMix syringes deliver bubble-free, perfectly mixed material that can be applied directly from the syringe. The **Bifix QM** system achieves excellent adhesion with suitable dentine/enamel bonds (**Futurabond DC**) and **Ceramic Bond** for silanisation of the restoration.

Indications:

Adhesive luting of inlays, onlays, veneers, crowns, partial crowns, bridges, root posts and adhesive bridges (Maryland bridges).

Method of use:

Please additionally consult the instructions for use of **Futurabond DC** and **Ceramic Bond** for detailed information and instructions.

1. Preparation

Remove the temporary if necessary. Clean, thoroughly rinse and slightly dry the tooth with air.

Note

The working area must be kept clean and dry. Use of a rubber dam is recommended. Teeth covered with a rubber dam may dry out, which can slightly influence the appearance of the shade.

2. Preliminary preparation

Protect dentine in close proximity to the pulp with a suitable lining, e.g. calcium hydroxide and/or glass ionomer cement.

3. Conditioning the restoration**a. Full ceramic restorations**

The inner adhesive surfaces of the restoration should be micro-etched with a hydrofluoric acid etchant. Normally, the dental laboratory will already have done this.

Etch the restoration for 15 s with a phosphoric acid gel, then rinse and dry.

Place **Ceramic Bond** (silane) in a mixing palette. Apply **Ceramic Bond** to the inside of the restoration with a brush and allow it to take effect for 60 s. Afterwards, dry for 5 s with oil-free air. Do not touch or otherwise contaminate the conditioned surfaces.

b. Metal, composite and zirconium dioxide restorations

Sandblast the surfaces with a maximum of 1.5 - 2 bar pressure and a suitable blasting agent. Follow the manufacturer's instructions for use for the respective material.

Carefully remove the blasting abrasive residue and dry the adhesive surfaces. Apply **Ceramic Bond** to the conditioned surfaces with a brush, allow to take effect for 60 s and then carefully dry with oil-free air. Do not touch the adhesive surfaces afterwards and quickly lute the restoration with **Bifix QM** while observing the following points.

c. Chairside luting of inlays

Clean the bonding surface with alcohol and dry with oil-free air. Do not touch the adhesive surface thereafter – insert and bond the restoration in accordance with the following points.

4. Application of the bonding

Use **Bifix QM** in conjunction with a suitable bonding agent. Follow the instructions for use of **Futurabond DC**.

5. Luting the restoration with Bifix QM

Insulate/block out any removable prosthodontic elements around the connectors before luting. Only mix **Bifix QM** immediately before luting, as described in the following: Due to technical reasons there may be slight differences in the filling levels of the two syringe chambers prior to first use. Therefore remove syringe cap and extract material until both chambers deliver uniformly. Attach a mixing tip type 11. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the cannula into place. The material is automatically mixed in the mixing tip and can be applied directly onto the prepared contact areas. If required, the optional intraoral tip type 4 can be placed onto the mixing tip.

Working time in reduced light is approximately 2.5 - 3.5 min. Setting time is 3 min from the time of inserting the restorative piece.

Insert the piece and fix it in place with slight pressure. Remove excess amounts of **Bifix QM** with a foam pellet or a disposable brush. Dental floss is recommended for removing excess material from interdental spaces, leave a small amount of excess material in place – see below.

Advice for luting Maryland bridges

When luting Maryland bridges and using **Futurabond DC** to bond, the enamel must either be roughened or conditioned with a phosphoric acid etching gel before applying the bonding agent.

6. Luting of fibre-reinforced resin posts (in combination with Futurabond DC):

- When luting root posts, massage the mixed **Futurabond DC** (liquid 1 and 2) into the root canal and into the occlusal surface of the core for 20 s with an applicator. Disperse solvent with oil-free air for 5 s. Remove any excess material from the canal with an absorbent strip of paper. Do **not** light-cure the bonding layer.
- It is recommended to apply **Bifix QM** directly into the root canal using fine intraoral tips type 1. Do not use a lentulo! After preparing the root post according to the manufacturer's instructions, it can additionally be moistened with **Bifix QM**. Insert the post making sure to leave a small amount of excess material.
- To fix the post and cure the **Futurabond DC** applied to the occlusal surface, light-cure for at least 40 s.

7. Light-curing

The polymerisation time can be controlled by additional light-curing. Light-cure the material at the luting margins with a halogen or LED light for a minimum of 20 s per segment. The light output should be a minimum of 500 mW/cm². Hold the tip of the light guide as close as possible to the surface to be cured. The restorative piece must be fixed in its desired position with a suitable instrument during light-curing.

Advice on oxygen inhibition

Like all resin-based materials, **Bifix QM** surfaces in contact with oxygen do not cure completely (approximately 50 µm inhibited layer). It is therefore recommended to use one of the two following methods:

- Leave a small amount of excess material and remove it during the subsequent finishing procedure.
- Remove the excess material completely and cover the margins of the restorative piece with oxygen-impermeable protective gel.

8. Finishing/Fluoridation

The removal of excess amounts is possible immediately after polymerisation. Remove cured excess material with a finishing diamond or flexible polishing disk. Finish approximate areas with finishing and polishing strips. Enamel surfaces which have been excessively etched or damaged during finishing should then be fluoridated with a fluoride varnish.

Information, precautionary measures:**Only use suitable bonding materials.**

Some enamel/dentine bonding agents are not compatible with dual-curing composites. When in doubt, consult the manufacturer's instructions for use for the respective bond. For best results and an optimal bond, use **Futurabond DC**.

Bifix QM contains Bis-GMA, benzoyl peroxide and amines. **Bifix QM** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

In case of contact with the eyes, rinse immediately with plenty of clean water and seek medical advice.

Irritation to the pulp may occur if dentine thickness is insufficient (less than 1 mm). In such cases, take appropriate measures to protect the pulp.

Residues from preparations containing thymol, eugenol or oil of cloves disturb the curing process of the luting composite and therefore should be avoided. Use only eugenol-free temporary cements for temporary luting.

Bifix QM is not suitable for fixing brackets due to its high adhesion values. The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Storage:

Store at temperatures of 4 °C to 23 °C. Do not use after expiry date.

Refrigeration is recommended.

Allow **Bifix QM** to reach room temperature before use.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1210 E1 0817 99 © by VOCO