Carefully read the instructions prior to use

Product description:
Direct application dispenser for restorative material in caps (composite application system).
The parallel lever movement allows a simple, precise application, of even highly filled composites.


Patient target group:
Caps Dispenser is suitable for use in all patients without any age or gender restrictions.

Performance features:
The product's performance features satisfy the requirements of the intended use and the relevant product standards.


User:
Caps Dispenser should only be used by a professionally trained dental practitioner.

User


- Pull the quick-release fastener all the way back.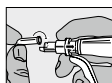


- Insert the cap into the dispenser and slowly release the fastener until the cap is securely locked into place.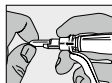

- Rotate the tip of the cap to the desired direction and remove the protective cap. Extrude the material by slowly and evenly pressing the handle and lever together, applying the material directly into the cavity. Do not apply too much pressure.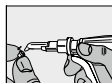

- After the material has been applied, pull the quick-release fastener back and pull the cap forward and up to remove it.

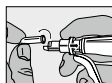- Clean and disinfect the **Caps Dispenser** after use. The **Caps Dispenser** can be steam-sterilized up to 134°C (steam autoclave, follow the manufacturer's instructions for use). In addition, ethanol and propanol can be used for cleaning.

Warnings, precautionary measures:

- The **Caps Dispenser** system allows quick and hygienic direct application. The material can be precisely dispensed into the cavity through the small opening of the cap. Detailed information for each of the **VOCO** preparations can be found in the individual instructions for use. Do not use visibly damaged devices.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Disposal:
Dispose of the product in accordance with local regulations.

Reporting obligation:
Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Caps Dispenser** must be reported to VOCO GmbH and the responsible authority.

PRODUCT ORDERING INFORMATION:

Caps Dispenser REF 9301

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. VOCO recognizes its responsibility to replace products if proven to be defective. VOCO does not accept liability for any damage or loss, directly or indirectly, stemming from the use of inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith. Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.
For questions or comments, please call 1-888-658-2584.

Keep this material out of reach of children.
For dental use only.
An explanation of the symbols used in labeling can be found at www.voco.dental/symbols

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