D Gebrauchsinformation

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GB Instruction for use

DC Fissureseal is a light-curing composite with fluoride for fissure sealing. The main component is the oligomeric dimethacrylate bis-GMA. It is used to seal fissures and occlusal surfaces for the purpose of caries prophylaxis.

Application

1. Thoroughly clean the surface to be sealed and coat with etching gel (orthophosphoric acid). The blue colour makes it easy to apply the gel accurately and uniformly. Allow the etching gel to act for approx. 60 seconds. Shorten the etching time for the enamel of teeth of young people to approx. 30 seconds, and extend to approx. 120 seconds for prism-deficient or fluoridised enamel. Rinse thoroughly with plenty of water and then dry thoroughly the surface that has to be sealed well. The etched tooth surfaces should have a milky-white appearance. After drying avoid contact with liquids.

2. Apply a single drop of DC Fissureseal on a mixing pad and brush a thin layer on the etched surfaces. Cure for 20- 30 second with a halogen lamp. After use tightly reclose the syringe.

Side-effects

Provided that this medical product is correctly processed and applied, undesirable side-effects are very rare. In the event that undesirable sideeffects do appear - even when in doubt - please inform us and give a precise description of the accompanying circumstances and symptoms. We will definitely follow up all notifications.

Contra-Indication

Allergies against acrylates.

Application remarks

The application instructions and other precautionary measures described in the directions for the use of this product must be observed. If patients still develop undesirable reactions or other effects, even though these instructions have been carefully observed, please notify us in all cases and give a precise description of the accompanying circumstances.

Package

DC Fissureseal, white, 2 x 2.0 ml syringes
+ 1 x 2.0 ml etching-gel
+ tips

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

Safety instructions:

H315: Causes skin irritation.
H319: Causes serious eye irritation.

P264: Wash thoroughly after handling.
P280: Wear protective gloves/protective clothing/eye protection/face protection.
P302+P352: IF ON SKIN: Wash with plenty of water and soap.
P321: Remove mechanically and wash skin thoroughly with soap and water.
P332+P313: If skin irritation occurs: Get medical advice / attention.
P362+P364: Take off contaminated clothing and wash it before reuse.
P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P337+P313: If eye irritation persists get medical advice/attention.

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DC Fissureseal, blanc 2 x 2,0 ml
+ 1 x 2,0 ml Extra-Gel + canules

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