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FANTESTIC® DirectCem DC

CE 0482

Hersteller

R-dental Dentalerzeugnisse GmbH - Winterhuder Weg 88 - D-22085 Hamburg
 Telefon +49 (0)40 - 22 75 76 17 - Fax +49 (0)40 - 22 75 76 18 - www.r-dental.com

FANTESTIC® DirectCem DC (KwikkMix® -System 4:1)**Bestell-Nr.: FDS4262 (Transluzent)****Bestell-Nr.: FDS4265 (A2 Universal)****Beschreibung und Indikation**

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Order-No.: FDS4265 (A2 Universal)

Description and indication

FANTESTIC® DirectCem DC is a self-adhesive, self-curing luting cement with light-curing option (dual-cure, DC) for definite self-adhesive luting ("cementation") of

• indirect restorations such as crowns and bridges made of metal, metal-ceramic, all-ceramic (e. g. zirconium dioxide), CAD/CAM-manufactured ceramic and

• of inlays, onlays made of ceramic, metal, composite and posts and orthodontic materials made of fibre reinforced composite (e. g. pre-silanized glass fibre posts, such as FANTESTIC® POST).

FANTESTIC® DirectCem DC is further indicated for definite self-adhesive luting of restorations made of metal, metal ceramic and all-ceramic (e. g. zirconium dioxide) on implant abutments.

There is *no* need to prime and bond teeth before luting.

Restorations made of glass-ceramic and fibre reinforced composites have to be silanized with FANTESTIC® CerBond before luting (see section 4).

Restorations made of metal, metal-ceramics, all-ceramics (e. g. zirconium dioxide) must be primed with the universal primer ZM-Bond® (see section 4).

Contraindications

FANTESTIC® DirectCem DC is *not* indicated for thin ceramic restorations made of feldspathic ceramics or low fusing ceramics, veneers and Maryland-bridges.

Areas near the pulp must be protected but *not* with the luting cement FANTESTIC® DirectCem DC (see section 2 a.).

Characteristics and composition

FANTESTIC® DirectCem DC is based on methacrylate resin and inorganic fillers. DirectCem DC has a filler content of about 50 % by weight and 45 % by volume.

FANTESTIC® DirectCem DC neutralizes itself after application.

FANTESTIC® DirectCem DC is delivered in a 5 ml-double chamber syringe in the safety cartridge system (KwikMix® -System 4:1).

Working time and hardening time

Working time and hardening time of FANTESTIC® DirectCem DC depends on temperature. Immediately after automatically mixing:

Extraoral application at about 23 °C ± 1 °C until insertion intraoral at about 37 °C ± 1 °C:

Working time	1 min 30 s ± 10 s
Hardening time (without working time)	4 min s ± 10 s

Recommended procedures

1. Isolation

A dry area and use of rubber dam (Order-No.: KDL31XX, XX = depends on strength) or latex-free dam (Order-No.: KDL3500) is recommended.

2 a. Preparation

Prepare in accordance with the clinical rules, and in case the pulp should be protected with a Ca(OH)₂ cement (for example Cal de Luxe®, Order-No.: CDL2503).

2 b. Preparation of the root canal

Prepare the root canal in accordance with the clinical rules e. g. with an sodium hypochloride solution, approx. 3 %, rinse with water and dry with paper points.

3. Preparation of FANTESTIC® DirectCem DC 5 ml-double chamber syringe (KwikMix® -System 4:1)

The double chamber syringe will be removed from the seal-pack and the date of removal should be documented on the seal-pack. Use FANTESTIC® DirectCem DC after opening within 6 months and before the end of expiration date.

Before first use screw off the original cap and dispose it. Before screwing on a new mixing cannula the double chamber syringe should be tested if both pastes are extruded equally.

FANTESTIC® DirectCem DC (KwikMix®) will be mixed light protected and automatically using the MIXING CANNULA MPN (Order-No.: MIK1045) or using the MIXING CANNULA MPN 4 (Order-No.: MIK1048) + FANTESTIC® Root-Canal-Tip (Order-No.: RCT2041) of the syringe system. By using the fine Root-Canal-Tip the luting cement can be inserted directly and efficiently into the root canal.

4. Pretreatments

4 a. Pretreatments of indirect restorations or frameworks

In front of an adhesive luting glass-ceramic restorations must be silanized (FANTESTIC® CerBond, Order-No.: FCF4100).

Restorations made of zirconium dioxide, metal (precious, non-precious), metal-supported restorations and ceramic restorations must be treated in front of an adhesive luting with the universal primer ZM-Bond® (Order-No.: ZMF4200). ZM-Bond® is a single-component primer, which promotes a chemical adhesion between luting cements and restorative materials based on metal, zirconium dioxide and ceramic.

ZM-Bond® has chemical groups, which on the one hand include functional groups on methacrylate basis for luting cements and on the other hand functional groups for an adhesion to indirect restorative materials and frameworks.

Metal

The adhesive contact areas of the restoration will be sandblasted according to the instructions for use of the manufacturers, until a matt surface has been achieved. Then rinse the contact areas with water or clean it in an ultrasonic device and dry with oil-free air.

Attention: Do not use phosphoric acid on the adhesive contact areas!

ZM-Bond® should then be applied with a FANTESTIC® MICRO BRUSH (Order-No.: MPB3981) onto the pre-treated surfaces of the restorations. The primer should be effect for 10 s and then dispense the solvent with oil-free air.

Zirconium dioxide

The adhesive contact areas of the restorations should be prepared according to the instructions for use of the manufacturers. Then rinse the contact areas with water or clean it in an ultrasonic device and dry with oil-free air.

Attention: Do not use phosphoric acid on the adhesive contact areas!

ZM-Bond® should then be applied with a FANTESTIC® MICRO BRUSH (Order-No.: MPB3981) onto the pre-treated surfaces of the restorations. The primer should be effect for 10 s and then dispense the solvent with oil-free air.

Ceramic

The adhesive contact areas should be etched with hydrofluoric acid (HF) according to the instructions for use of the manufacturers. Then rinse the prepared contact areas with water and dry with oil-free air.

ZM-Bond® should then be applied with a FANTESTIC® MICRO BRUSH (Order-No.: MPB3981) onto the pre-treated surfaces of the restorations. The primer should be effect for 10 s and then dispense the solvent with oil-free air.

4 b. Pretreatments of tooth structure

Priming the tooth structure is *not* required.

5. Luting of restorations with FANTESTIC® DirectCem DC

FANTESTIC® DirectCem DC should be mixed light protected and automatically using the MIXING CANNULAS MPN (Order-No.: MIK1045) of the safety cartridge system.

Before application a small amount of material (size like rice corn) should be pressed out and should be disposed of immediately.

For an adhesive luting self-curing or dual-curing is the main process. Apply the automatically mixed FANTESTIC® DirectCem DC directly within 1 min 30 s (working time) in a thin layer of about 0,5 mm onto the primed areas of the restoration and place it with slight pressure and hold.

Excess material of FANTESTIC® DirectCem DC can be removed by using brushes, FANTESTIC® MICRO BRUSH (Order-No.: MPB3981), dental floss or scaler. The excess material can be removed easily in time if polymerized from all sides for few seconds with light.

FANTESTIC® DirectCem DC will polymerize within about 4 min (99 °F, 37 °C), independent of light-curing. Optionally it can be light-cured additionally.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap.

The double chamber syringe should be stored dark within the seal-pack.

6. Light-cure, oxygen inhibition layer and work out

The restoration margins should be light-cured for about 20-30 s after removal of excess material.

In order to avoid the oxygen inhibition layer (about 100 µm) the water soluble FANTESTIC® OxyBlok (Order-No.: FOS3970) should be applied on the restoration margins immediately after removal of excess material. After final polymerization OxyBlok can be rinsed off with water.

Then the restoration margins could be polished.

7. Cementation of posts

For self-adhesive cementation of glass fibre posts or fibre reinforced composite (FANTESTIC® POST, Order-No.: FPS2X1X, X depends on geometry and size) the self-cure or dual-cure mode is valid.

The root canal will be prepared „lege artis“ according to section 2 b.

FANTESTIC® DirectCem DC should be mixed light protected and automatically using the MIXING CANNULA MPN (Order-No.: MIK1045) or using the MIXING CANNULA MPN 4 (Order-No.: MIK1048) + FANTESTIC® Root-Canal-Tips (Order-No.: RCT2041) of the safety syringe system. By using the fine Root-Canal-Tip the luting cement can be inserted directly and efficiently into the root canal.

Before application a small amount of material (size like rice corn) should be pressed out and should be disposed of immediately.

FANTESTIC® DirectCem DC should then be luted directly into the root canal within the working time of 1 min 30 s. Start filling from apical as deep as possible and maintain the tip in the cement to avoid bubbles. Remove the tip out of cement when the canal has been filled.

Optional FANTESTIC® DirectCem DC can be applied onto the post (it can be silanized using FANTESTIC® CerBond, Order-No.: FCF4100 or ZM-Bond®, Order-No.: ZMF4200) and then will immediately be screwed into the root canal within the working time of about 1 min 30 s and held with slight finger pressure.

It will self-cure after working time within about 4 min. The coronal areas should be light-cured from each side for about 20 s.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap.

The double chamber syringe should be stored dark within the seal-pack.

Precautions to application, product life and/or storage

Pay attention to the material safety data sheets (MSDS >> www.r-dental.com).

Do not use FANTESTIC® DirectCem DC in case of allergic reactions (e. g. methacrylates). Areas near the pulp must be protected but *not* with the luting cement FANTESTIC® DirectCem DC (see section 2 a.).

Avoid contact of FANTESTIC® DirectCem DC (especially the catalyst paste) with soft tissues, skin or eyes. A contact with eyes may cause serious damage to eyes, so always wear eye protection. If there is an accidental contact with eyes, flush immediately with copious amounts of water and consult a physician.

Do not adjust the viscosity by using any resin.

Do not store FANTESTIC® DirectCem DC in proximity of eugenol containing materials nor let it come in contact with eugenol containing materials. Eugenol can impair hardening and cause discoloration. Further on no oxidic disinfection agents (like H₂O₂) should be used, because these can inhibit hardening.

The double chamber syringe will be removed from the seal-pack and the date of removal will be documented on the seal-pack (use FANTESTIC® DirectCem DC after opening within 6 months and before the end of expiration date).

Before first use screw off the original cap and dispose it. Before screwing on a new mixing cannula the double chamber syringe should be tested if both pastes are extruded equally.

Before application a small amount of material (size like rice corn) should be pressed out and should be disposed of immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the cartridge cap.

Expiration date see label. Do not use after expiration date.

FANTESTIC® DirectCem DC should be stored between 10-25 °C (50-77 °F) and horizontally, that is laying dark within the seal-pack. Use FANTESTIC® DirectCem DC after opening within 6 months and before the end of expiration date. Avoid a constantly high humidity. Do *not* store FANTESTIC® DirectCem DC in a refrigerator. Keep out of the reach of children!

FANTESTIC, Cal de Luxe, ZM-Bond and KwikkMix are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for deflections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

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© 2013 R-dental Dentalerzeugnisse GmbH. All rights reserved.
© 2013 R-dental Dentalerzeugnisse GmbH. Todos los derechos reservados.