**VOCO****Grandio® blocs****Instructions for use****Product description:**

**Grandio blocs** is a nano-hybrid composite CAD/CAM block for the fabrication of permanent single-tooth restorations.

**Grandio blocs** contains 86 % w/w inorganic fillers in a polymer matrix. The restorations produced must be placed with an adhesive composite luting agent in combination with a dentine-enamel bonding agent. **Grandio blocs** is available in two sizes (12 and 14L).

**Shades (two degrees of translucency):**

High Translucent (HT): A1, A2, A3, A3.5  
Low Translucent (LT): A1, A2, A3, A3.5, B1, C2, BL

**Indications:**

- Crowns, inlays, onlays, veneers
- Implant-supported crowns

**Shade selection:**

Use the VITA® shade system (**GrandioSO**) to determine the shade against the cleaned but unprepared tooth prior to anaesthesia and preferably in daylight conditions.

**Core and cavity preparation:**

As a general rule, the core/cavity preparation should be performed in accordance with the conventional rules for all-ceramic restorations. That means rounding off interior corners and edges and using a shoulder preparation with rounded interior edges or a chamfer preparation. The temporary restoration and any contamination should be mechanically removed (e.g. using pumice and a polishing brush) and the area subsequently rinsed with water. Observe the minimum thicknesses stated below.

**Note:** Do not create any undercuts.

**Crowns, inlays and onlays (see Fig. 1 - 3):**

- Preparation angle of 4° - 6°
- Minimum occlusal wall thickness 1.5 mm
- Minimum buccal wall thickness 0.8 mm
- Minimum wall thickness below load-bearing cusp 1.5 mm
- Minimum cervical wall thickness 0.8 mm

**Veneers (see Fig. 4):**

- Minimum cervical wall thickness 0.4 mm
- Minimum labial wall thickness 0.5 mm
- Minimum incisal wall thickness 0.5 mm

**Cement space:**

70 µm (± 10 µm)

**Milling process:**

Select the correct block size for the planned restoration as well as the milling parameters for **Grandio blocs** or for composites. When doing so, pay attention to the software settings of the respective CAD/CAM systems. First of all, the settings for block sizes and the milling parameters need to be created, if they are not already included in the software settings of the CAD/CAM systems. Please contact the CAD/CAM system provider for assistance. The operating instructions from the manufacturers must be observed and followed.

Check the processed restoration for defects such as fissures or material chipping. If the restoration displays any defects, it must be rejected.

**Note: Grandio blocs is an already completely cured composite and must not be fired.**

**Polishing:**

The restoration can be finished and polished with standard composite polishers (e.g. medium, fine or extra fine diamond grinding tools, polishers), this can be done either intraorally or extraorally.

**Adhesive luting:****Preparing the restoration:**

For an optimal bond, blast the luting surface of the restoration with aluminium oxide (25 µm - 50 µm) at 1.5 bar - 2 bar. Use a suction device to remove the dust produced.

**Note:** Do not etch with hydrofluoric acid or phosphoric acid.

Clean the pretreated restoration using an ultrasonic bath or steam cleaner and then dry with oil-free air. Final cleaning with medical alcohol is possible. Treat the luting surface with a silane coupling agent such as **Ceramic Bond**. Apply **Ceramic Bond** to the inside of the work and allow it to work for 60 seconds. Then dry for 5 seconds with oil-free air. The instructions for use must be observed and followed.

**Adhesive luting of the restoration:**

The processed restoration must be luted with a composite-based, adhesive luting agent. The instructions for use must be observed and followed. **VOCO** recommends the dual-curing luting system **Bifix QM** in combination with **Ceramic Bond** for an optimal bond to the restoration and a suitable dentine-enamel bond (**Futurabond DC/Futurabond U**). Alternatively, it is also possible to use a purely light-cured, flowable composite/ORMOCER® (e.g., **GrandioSO Flow**) if the restoration has thin walls (≤ 2 mm).

**Modifications:**

**Grandio blocs** is a nanohybrid composite CAD/CAM restorative material which can be modified, characterised or repaired at any time using a composite/ORMOCER®. Roughen the surface of the restoration (for example with a diamond bur) and apply a suitable adhesive system (e.g. **Futurabond U**) in accordance with the instructions for use. Then apply the composite/ORMOCER® (e.g., **GrandioSO, GrandioSO Flow, FinalTouch**) in accordance with the respective instructions for use.

**Information, precautionary measures:**

- No known side effects. However, the sensitisation of hypersensitive patients cannot be ruled out.
- **Grandio blocs** contains methacrylates. **Grandio blocs** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

**Storage:**

Store at temperatures of 4 °C to 23 °C. If the material is stored in the refrigerator, allow it to reach room temperature before use. Do not store the product in a place where it is exposed to direct artificial light or sunlight. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA6000 E1 0118 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VITA® is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co.
**VOCO GmbH**