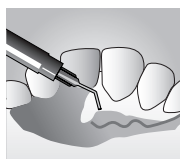
# smartrepair® rosa

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Aguja de aplicación 02714  
10 pz.

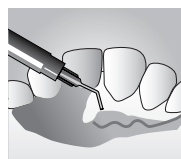
# smartrepair<sup>®</sup> rosa

Resin-based, light-curing model and repair material

## 1. Preparation

Thoroughly rinse and dry the temporary restoration or denture. Roughen and clean all other areas to be treated and completely remove all grinding dust. Apply **smartbond<sup>®</sup>** extraorally with smartbrush covering the whole area and rub in, carefully blow away excess with air syringe and allow to dry for approx. 1 min. When reattaching teeth or fabricating immediate dentures illuminate the areas to be treated and prepare notches or grooves in order to ensure sufficient mechanical retention to the plastic.

## 2. Application

Remove seal and fit the enclosed application needle. These needles enable the material to be applied with pinpoint accuracy and are intended for single use only. Reseal the application syringe after use. Consider an intermediate polymerisation step for layer thicknesses over 2 mm and apply the material in two or more layers and cure. To ensure optimum adhesion of the layers do not remove the surface inhibition layer during application. Do not wipe the surface with isopropanol until application has been completed.

## 3. Curing

**smartrepair<sup>®</sup> rosa** is polymerised by light at a wavelength range of between 360 to 480 nm and can be cured with standard devices within the specified time.

**In the practice:** use a handheld lamp for approx. 40 sec. per surface.

**In the laboratory:** use halogen polymerisation devices for approx. 10 min.,  
UVA tubes for approx. 15 to 20 min.,  
strobe light for approx. 3 minutes.

## Important information

- Use alginate insulation when working on the plaster model.
- Use a suction system when trimming and polishing.
- The constituents of **smartrepair<sup>®</sup> rosa** can trigger allergic reactions in patients who are predisposed to allergies. Wear suitable eye protection and gloves! In such a case further use of the product should be discontinued.
- Avoid contact with the skin and eyes. If accidental contact occurs, rinse under running water and seek medical assistance if necessary.
- Seal bottle carefully immediately after use. Remove the cannula and use the original cap. Use a new application needle for the next treatment.
- Store in a dark place.
- To be used and applied only by qualified personnel.
- Please follow the instructions of the safety data sheet!

# DETAX

GmbH & Co. KG  
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany  
Telefon: 07243/510-0 · Fax: 07243/510-100  
www.detax.de · post@detax.de

Made in  
Germany

04/2016

GB

Instructions for use

## Indications for use:

### In the practice:

- Aesthetic adjustment and correction of the gingival areas of temporary crowns and bridge restorations.
- Chairside repairs of acrylic resin based dentures and correction of air voids, cracks and pores in these dentures
- Reattachment of teeth to dentures

### In the laboratory:

- Repairs of fractures and denture bases
- Denture base extension and contouring
- Completing of partial dentures and relinings
- Fabrications of small single tooth immediate dentures

## Technical data:

- **Volume:** 3 g syringe
- **Dosage and application:** direct application by application syringe
- **Product colour:** transparent pink
- **Curing:** By light at a wave length of 360 to 480 nm
- **Application:** At 23°C ± 2°C
- **Storage:**

Protect from heat and exposure to sunlight

## Ordering information:

**smartrepair<sup>®</sup> rosa**  
Standard pack pink 03364  
Application syringe 3 g  
Application needles 8 pcs.

**smartrepair<sup>®</sup>**  
**smartrepair<sup>®</sup> set** 02694  
Application syringe  
A2 1,5 g  
Application syringe  
A3.5 1,5 g  
smartbond<sup>®</sup> bottle 5 ml  
smartbrush 5 pcs.  
Application needles 6 pcs.  
Flowchart 1 pc.

**Standard pack**  
Application syringe 1,5 g  
Application needles 5 pcs.  
A2 02772  
A3 02862  
A3.5 02695

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