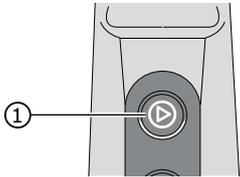
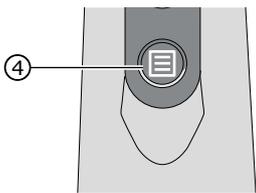
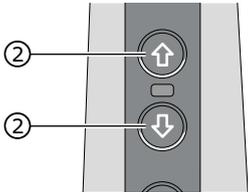
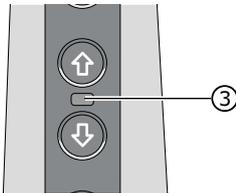


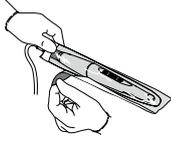
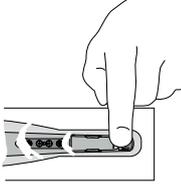
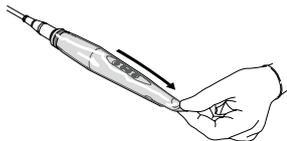
Quick Reference - DIAGNOcam Vision Full HD

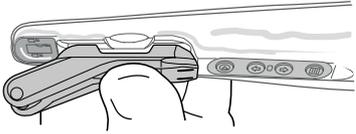
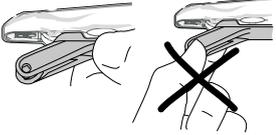


DIAGNOcam Vision Full HD		
Installation / Setup		► For software installation and start up note the instructions for use and the installation instructions of the software.
Power on		The camera can be switched on by the following means: <ul style="list-style-type: none"> • System start of the imaging software • Pressing any button on the control panel • Removal of the camera from the table holder or wall mount
Capture an image Start / stop a video		► Press the trigger button ① for capturing an image and/or for starting/stopping the video recording.
Switch between modes		► To switch between the modes (intraoral, fluorescence and transillumination), press the menu button ④.
Navigation of tooth chart		► Press the buttons ② for navigation in the numbered tooth chart (if available in your software).
Status LED		Status LED ③ indicating the status of the camera: <ul style="list-style-type: none"> • Withe = Camera is active • Blue = Standby mode • Red = Error
Power off		The camera can be switched off by the following means: <ul style="list-style-type: none"> • Placing the camera in the table holder or wall mount • Extended period of inactivity

Quick Reference - DIAGNOcam Vision Full HD



Using the sheath		
Attach the sheath		▶ Slowly apply the sheath over the camera to allow the air to escape.
		▶ Remove the outer protective paper.
		▶ Tighten the clear view window in front of the exposure window of the camera. ▶ The clear view window of the sheath must be situated exactly over the exposure window of the camera.
Remove the sheath		▶ Carefully remove the sheath after each treatment and dispose of it.

Snap-in and detach the tip		
Snap-in the tip		▶ Position the tip on one side of the housing and carefully snap-in into the opposite side. ▶ Check on the stable positioning of the tip.
Detach the tip		▶ Detach the tip carefully by rotating it on one side, then press off the tip at the predetermined recess and do not touch it by the soft silicone legs. ▶ After every use process the tip in accordance with the instructions for use.

Attach and detach the fluorescence cover		
Attach the fluorescence cover		▶ Position the fluorescence cover on the housing. ▶ Check the stable positioning of the fluorescence cover.
Detach the fluorescence cover		▶ Detach the fluorescence cover carefully. ▶ After every use process the fluorescence cover in accordance with the instructions for use.